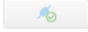


For PMA.core :

Host: Username: Password: Folder: 

status: Trying to connect...
status: Resolving address of
status: Connecting to
status: PMA.core v2.0.0.1686
status: Connection established...
status: Loading folder on PMA.core...
status: Folder successfully loaded...

- Host : PMA.core instance to connect to, it should be in this form "http(s)://xxxxx/pma.core.2/"
- Username & Password : connection credentials
- Folder : Remote directory on server to navigate to if connection is successful

For My Pathomation :

Email: Password: Folder: 

status: Trying to connect to the cloud...
status: Resolving address of
status: Connecting to
status: PMA.core v2.0.0.1696
status: Connection established...
status: Loading user's folder on the cloud...
status: Folder successfully loaded...

- Email : The email address used to subscribe to My Pathomation (Pathomation cloud).
- Password : Password received on inbox once the subscription & payment are validated.
- Folder : disabled text field, end user can only connect to its own subscription folder.

From:
<https://docs.pathomation.com/PMA.transfer/> - **PMA.transfer**

Permanent link:
https://docs.pathomation.com/PMA.transfer/doku.php?id=connection_bar_logging_text_area

Last update: **2020/09/01 03:57**

