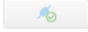


**For PMA.core :**

Host:  Username:  Password:  Folder:  

```

status: Trying to connect...
status: Resolving address of [redacted]
status: Connecting to [redacted]
status: PMA.core v2.0.0.1686
status: Connection established...
status: Loading folder on PMA.core...
status: Folder successfully loaded...

```

- Host : PMA.core instance to connect to, it should be in this form "http(s)://xxxxx/pma.core.2/"
- Username & Password : connection credentials
- Folder : Remote directory on server to navigate to if connection is successful

**For My Pathomation :**

Email:  Password:  Folder:  

```

status: Trying to connect to the cloud...
status: Resolving address of [redacted]
status: Connecting to [redacted]
status: PMA.core v2.0.0.1696
status: Connection established...
status: Loading user's folder on the cloud...
status: Folder successfully loaded...

```

- Email : The email address used to subscribe to Pathomation cloud.
- Password : Password received on inbox once the subscription & payment are validated.
- Folder : disabled text field, end user can only connect to its own subscription folder.

From: <https://docs.pathomation.com/PMA.transfer/> - PMA.transfer

Permanent link: [https://docs.pathomation.com/PMA.transfer/doku.php?id=connection\\_bar\\_logging\\_text\\_area&rev=1589595289](https://docs.pathomation.com/PMA.transfer/doku.php?id=connection_bar_logging_text_area&rev=1589595289)

Last update: 2020/05/16 05:14

