

Context menu for slides

Local site

	Slidename	Slidesize	FileType	Last Modified
	..			
	3D_FL_test01.mrxs	43,620,186	MRXS File	Thu Dec 26 06:34:59 GMT+01:00 2019
	clear.png	274	PNG File	Sun Dec 08 19:02:43 GMT+01:00 2019
	JP2K-33003-1.svs			05:40:22 GMT+01:00 2018
	JP2K-YCBCR-1.svs			06:36:50 GMT+01:00 2019
	sampleX.jpg			14:37:58 GMT+01:00 2019
	Test2.jpg			19:02:37 GMT+01:00 2019

Open slide in PMA.start

Upload slide to remote site


Inspect slideCtrl+Alt+I

Copy full pathCtrl+Alt+C

Initiated via mouse right click within the table view borders, it’s composed of four menu items : “Open slide in PMA.start”, “Upload slide to remote site”, “Inspect slide” and “Copy full path”.

Open slide in PMA.start :

Opens the slide in PMA.start

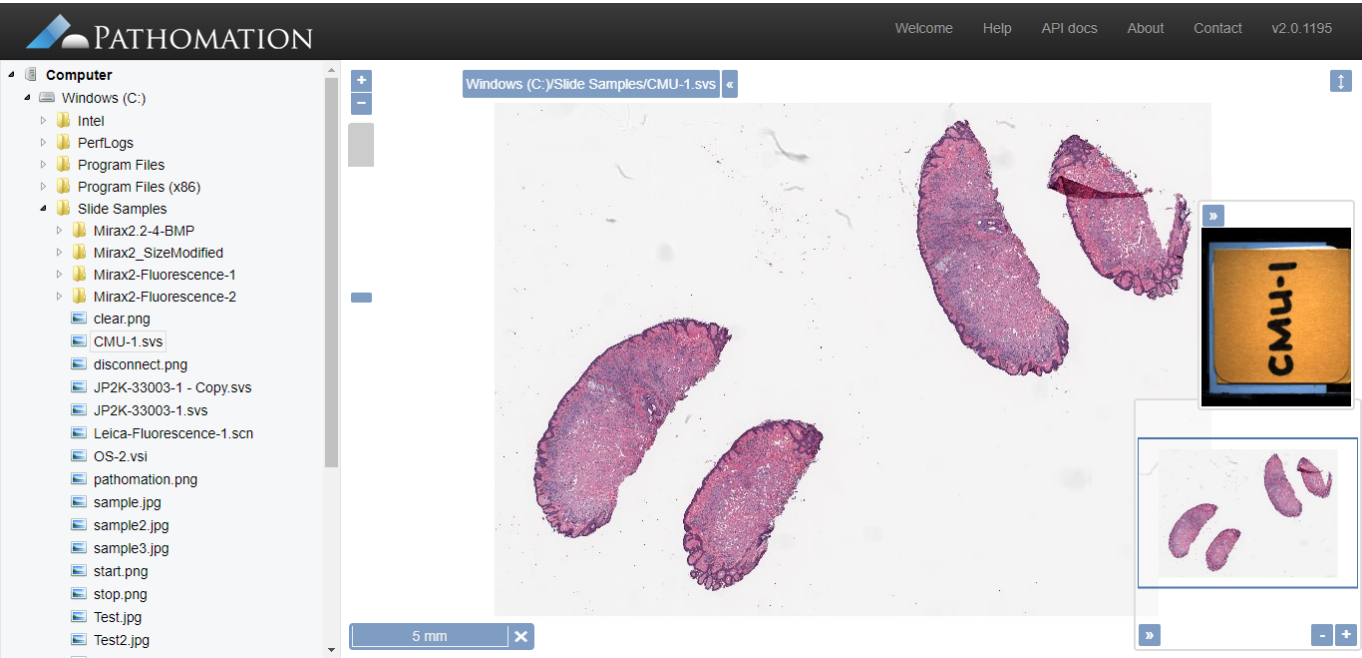


Upload slide to remote site :

Has been discussed in details earlier on this tutorial.

Inspect slide :

Information data regarding the slide (Thumbnail, fingerprint, size...)

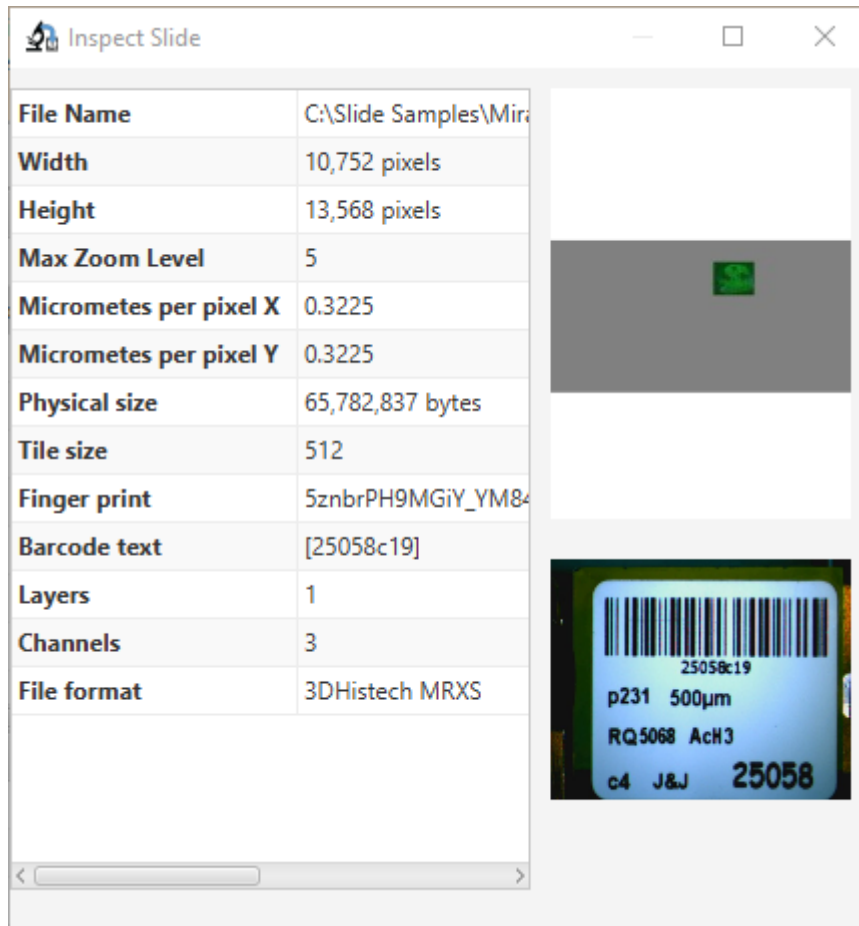

Copy full path :

Copies the full path of the local slide to the clipboard.

Remote site

	Slidename	Slidesize	FileType	Last Modified
↑	..			
📁	CMU-1		File folder	
📁	CMU-2		File folder	
📁	CMU-3		File folder	
📄	CMU-3.svs	253,815,733	SVS File	Sat Aug 03 20:34:02 WAT 2019
📄	CMU-1.svs		File	Fri Jul 19 15:57:26 WAT 2019
📄	CMU-1-JP2K-RGB.svs	132,565,343	SVS File	Fri Jul 19 16:02:51 WAT 2019
📄	CMU-2.svs	390,750,635	SVS File	Sat Aug 03 20:28:29 WAT 2019
📄	JP2K-YCBCR-1.svs	63,847,265	SVS File	Mon Jul 29 04:00:41 WAT 2019
📄	JP2K-YCBCR-2.svs	289,250,433	SVS File	Sat Aug 03 20:35:18 WAT 2019

Initiated via mouse right click within the table view borders, it's composed of one menu item :
"Transfer slide(s) to remote server".

Transfer slide to remote server :

Has been discussed in details earlier on this tutorial.

From:

<https://docs.pathomation.com/pma.transfer/> - **PMA.transfer**

Permanent link:

https://docs.pathomation.com/pma.transfer/doku.php?id=context_menu_for_slides

Last update: **2021/07/08 15:49**

