





In the context of PMA.transfer, a download refers to the action of transferring a slide from a remote site (either PMA.core or My Pathomation) to a local site (local HDD). There are four options to launch a download :

- Select a slide on remote site (Table view to the right) then click on button 
- Select a slide on local site (Table view to the left) then click on menu "Transfer" on the toolbar then select menu item "Download slide"
- Mouse right click inside the left table view borders, then select "Download slide to local folder" context menu item to download the slide to the local site.
- Drag & drop : drag a slide from the right, then drop it on the left. The cursor switches to  on drag,  on drop and  when an operation is forbidden.

#### Download of a folder (recursively) :

The download of a folder can be performed via a click on context menu item "download folder to local site" . Once successfully initiated, The folder and its sub-directories are sub-consequently created on local side then the download of contained slides is operated to corresponding destinations.

From:  
<https://docs.pathomation.com/PMA.transfer/> - **PMA.transfer**

Permanent link:  
[https://docs.pathomation.com/PMA.transfer/doku.php?id=initiating\\_a\\_download&rev=1599269619](https://docs.pathomation.com/PMA.transfer/doku.php?id=initiating_a_download&rev=1599269619)

Last update: **2020/09/05 04:33**

