





In the context of PMA.transfer, an upload refers to the action of transferring a slide from a local site (local HDD) to a remote server (either PMA.core or My Pathomation). There are four options to launch an upload :

- Select a slide on local site (Table view to the left) then click on button 
- Select a slide on local site (Table view to the left) then click on menu "Transfer" on the toolbar then select menu item "Upload slide"
- Mouse right click inside the left table view borders, then select "upload slide to remote slide" context menu item to upload the selected slide to the remote site.
- Drag & drop : drag a slide from the left, then drop it on the right. The cursor switches to  on drag,  on drop and  when an operation is forbidden.

#### Upload of a folder (recursively) :

The upload of a folder can be performed via a drag & drop or click on context menu item "upload folder to remote site" . Once successfully initiated, The folder and its sub-directories are sub-consequently created on server side then the upload of contained slides is operated to corresponding destinations.

From:  
<https://docs.pathomation.com/PMA.transfer/> - **PMA.transfer**

Permanent link:  
[https://docs.pathomation.com/PMA.transfer/doku.php?id=initiating\\_an\\_upload&rev=1599234065](https://docs.pathomation.com/PMA.transfer/doku.php?id=initiating_an_upload&rev=1599234065)

Last update: **2020/09/04 18:41**

