







## Windows


- Extract file PMA.transfer.2.2.0.103.zip into a destination folder
- Navigate to destination folder

|  |                   |                       |        |
|--|-------------------|-----------------------|--------|
|  msvcr100.dll     | 8/26/2020 1:59 AM | Application extension | 809 KB |
|  packager.dll     | 8/26/2020 1:59 AM | Application extension | 287 KB |
|  PMA.transfer.exe | 8/26/2020 1:59 AM | Application           | 115 KB |
|  PMA.transfer.ico | 8/26/2020 1:59 AM | Icon                  | 98 KB  |
|  ucrtbase.dll     | 8/26/2020 1:59 AM | Application extension | 979 KB |
|  vcruntime140.dll | 8/26/2020 1:59 AM | Application extension | 86 KB  |

- Double click on file PMA.transfer.exe

## MacOS

- Double click on file PMA.transfer.2.2.0.103.dmg
- The disk image gets mounted and you get prompted with the following window

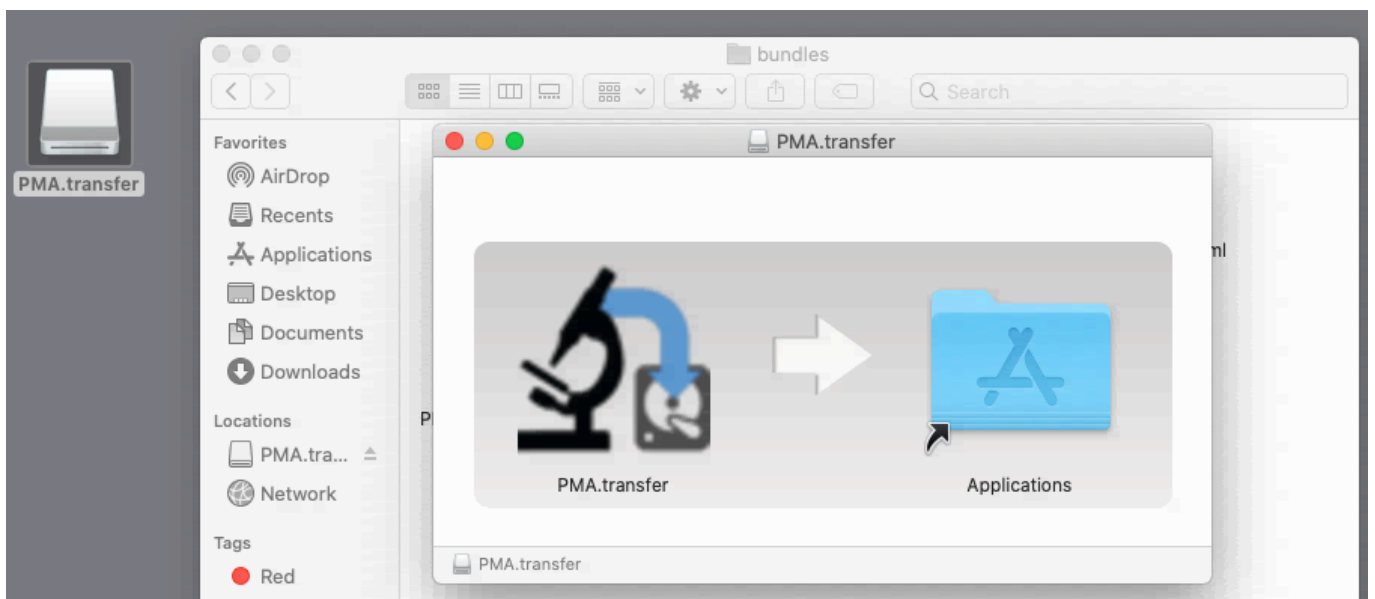

- Double click on PMA.transfer icon to launch the app

From: <https://docs.pathomation.com/PMA.transfer/> - PMA.transfer

Permanent link: <https://docs.pathomation.com/PMA.transfer/doku.php?id=installation&rev=1598921334>

Last update: 2020/09/01 03:48

