


Composed of three menu items : Details, Thumbnails, Refresh

Details :

This is the default view mode for the table view, it includes an icon, "Slidename", "Slidesize", "FileType", "Last Modified" columns

Thumbnails :

This view mode should be selected when it's needed to view the thumbnails of a slide (or a couple of slide). It includes an icon and a "Slidename" columns.

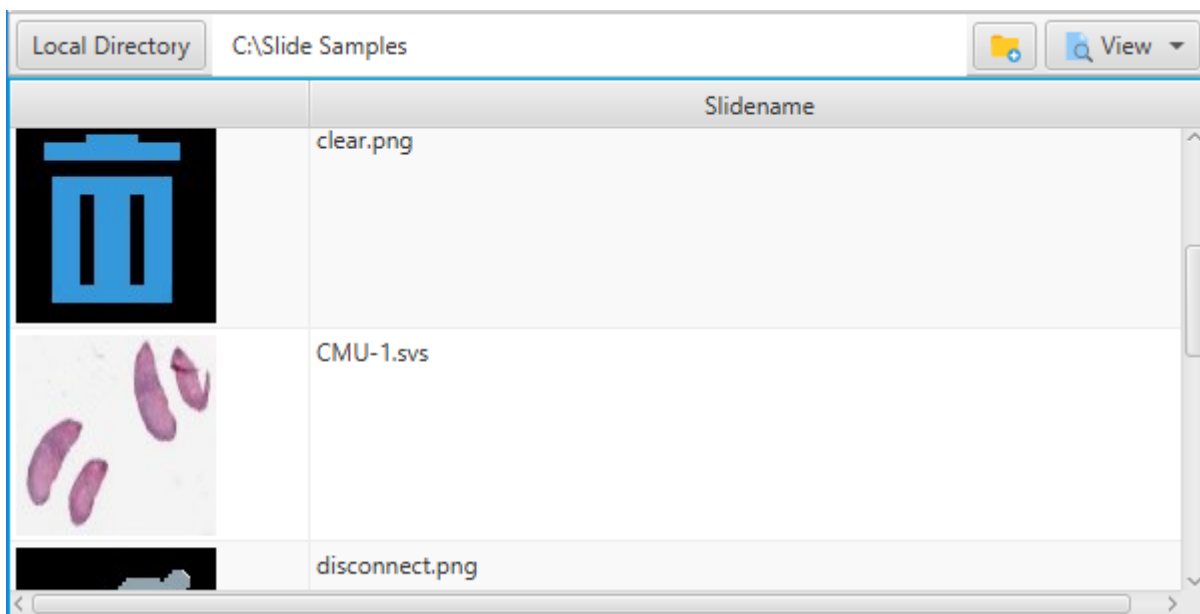

Refresh :

Refreshes the table view (it's worth noting that the table view, buttons "Add folder" and "View" get deactivated till the refresh is complete)

From: <https://docs.pathomation.com/PMA.transfer/> - **PMA.transfer**

Permanent link: [https://docs.pathomation.com/PMA.transfer/doku.php?id=menu\\_button\\_view&rev=1589595289](https://docs.pathomation.com/PMA.transfer/doku.php?id=menu_button_view&rev=1589595289)

Last update: **2020/05/16 05:14**

