


Menu button view (remote)

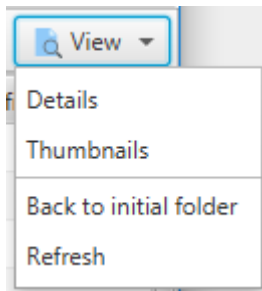

Composed of four menu items : Details, Thumbnails, Back to initial folder and Refresh.

Details :

This is the default view mode for the table view, it includes an icon, "Slidename", "Slidesize", "FileType", "Last Modified" columns

Thumbnails :

This view mode should be selected when it's needed to view the thumbnails of a slide (or a couple of slide). It includes an icon and a "Slidename" columns.

Back to initial folder :

This can be very useful to get back to initial folder of connection following series of navigation to parent/children folders.

Refresh :

Refreshes the table view (it's worth noting that the table view, buttons "Add folder" and "View" get deactivated till the refresh is complete)

From:
<https://docs.pathomation.com/PMA.transfer/> - **PMA.transfer**

Permanent link:
https://docs.pathomation.com/PMA.transfer/doku.php?id=menu_button_view_remote

Last update: **2021/07/08 15:50**

