

Release history

| Version | Revision | Released |
|------------------------------------|------------|----------------|
| PMA.transfer 2.0.0 | 50 | May 2019 |
| PMA.transfer 2.0.1 | 55 | July 2019 |
| PMA.transfer 2.0.2 | 60 | September 2019 |
| PMA.transfer 2.0.3 | 72 | January 2020 |
| PMA.transfer 2.1.0 | 85 | May 2020 |
| PMA.transfer 2.1.1 | 87 | June 2020 |
| PMA.transfer 2.2.0 | 103 | August 2020 |
| PMA.transfer 2.2.1 | 111 | February 2021 |
| PMA.transfer 2.2.2 | 114 | July 2021 |
| PMA.transfer 2.2.3 | 9d6e296f92 | August 2022 |
| PMA.transfer 2.2.4 | d67f7abd23 | February 2023 |
| PMA.transfer 2.2.5 | a08020023e | September 2025 |

What's new in version 2.0.1

General improvements :

- Log file menu item to open directly the current log file in addition to log files' location
- Added file format to “inspect slide” interface

Bug fixes :

- VSI slides' transfer issue
- Dealing with leading/trailing spaces in host & username text fields and trailing slashes for host as well

What's new in version 2.0.2

General improvements :

- Ask for confirmation before closing the app
- Auto detect white or black text color depending on text area background color
- Add recursive folder download feature

Bug fixes :

- Fix thumbnail URL encoding issue on local site
- Tests against all supported file formats, bug fixing

What's new in version 2.0.3

New features :

- Bypass proxy

- Delete directories on server side (only if empty and not a root one)
- Rename directories on server side (only if empty and not a root one)
- Delete slides on server side
- Rename slides on server side
- Add copy full path option both locally and remotely

General improvements :

- Directly download to selected folder on local site
- Keyboard input interaction
- New icon for root directories
- Better transfer speed metrics
- Better error messages
- Enhanced integrity check
- Adding counters to all bottom bar buttons

Bug fixes :

- Refresh table view for local site following successful downloads

What's new in version 2.1.0

New features :

- Interfacing with the new My Pathomation API
- Adding create/rename/delete folder features for My Pathomation accounts
- Adding rename/delete slide features for My Pathomation accounts
- Adding drag & drop functionality for download
- Release of a self-containing package for Windows (No installation of JRE required)

General improvements :

- Detection of PMA.core version on connection
- Harmonization of upload/download UI/UX
- Cleaning up UI for slides without barcode
- Defining explicit location while adding new folders

Bug fixes :

- Disabling of upload/download buttons on disconnection

What's new in version 2.1.1

General improvements :

- Compliance with PMA.start and PMA.start dotnetcore on Windows
- Adding support of VPN connections

What's new in version 2.2.0

New features :

- Server to server transfers
- Multiple selection of slides for transfer

General improvements :

- Compliance with PMA.start dotnetcore on MacOs & Unix
- Improved error messages (transfer of empty folders, transfers to location that require elevation of privileges..)
- Improved integrity check process

Bug fixes :

- Better handling of failed connections
- Fixes on the site manager

What's new in version 2.2.1

New features :

- User quota enforcement

General improvements :

- Error handling for uploads to Azure directories
- Improved error handling for server to server transfers
- Support for My Pathomation "add slide" API endpoint & "caller ID" parameter for slides uploaded via PMA.transfer
- Support for Keyboard key <Enter>

Bug fixes :

- Fixes on the server to server progress bar

What's new in version 2.2.2

New features :

- Improved reporting of successful slide transfers (interactive vs. console)

Bug fixes :

- Slide (file)size is reported correctly again
- Connectivity to My Pathomation is operational again

What's new in version 2.2.3

New features :

- Fill folders with slides and subfolders recursively
- Unload multiple selected slides
- Delete several selected slides
- New folder in the cloud S3 directory

Bug fixes :

- Recursion for delete, download and upload

What's new in version 2.2.4

General improvements :

- Upload and Download slides execute thru PMA-JAVA Maven dependency

New features :

- Renaming local folder
- Renaming local slide

Bug fixes :

- Progressive bar calculation in multi-component slides

What's new in version 2.2.5

General improvements :

- Fixed compatibility issues with Pma.Core.

New features :

- Added support for uploading slides larger than 5 GB to Azure and Amazon.

From:
<https://docs.pathomation.com/PMA.transfer/> - **PMA.transfer**

Permanent link:
https://docs.pathomation.com/PMA.transfer/doku.php?id=release_history&rev=1757661261

Last update: **2025/09/12 10:14**

