

# Release history

Version	Revision	Released
PMA.transfer 2.0.0	50	May 2019
<a href="#">PMA.transfer 2.0.1</a>	55	July 2019
<a href="#">PMA.transfer 2.0.2</a>	60	September 2019
<a href="#">PMA.transfer 2.0.3</a>	72	January 2020
<a href="#">PMA.transfer 2.1.0</a>	85	May 2020
<a href="#">PMA.transfer 2.1.1</a>	87	June 2020
<a href="#">PMA.transfer 2.2.0</a>	103	August 2020
<a href="#">PMA.transfer 2.2.1</a>	111	February 2021
<a href="#">PMA.transfer 2.2.2</a>	114	July 2021
<a href="#">PMA.transfer 2.2.3</a>	9d6e296f92	August 2022
<a href="#">PMA.transfer 2.2.4</a>	d67f7abd23	February 2023
<a href="#">PMA.transfer 2.2.5</a>	a08020023e	September 2025

## What's new in version 2.0.1

### General improvements :

- Log file menu item to open directly the current log file in addition to log files' location
- Added file format to “inspect slide” interface

### Bug fixes :

- VSI slides' transfer issue
- Dealing with leading/trailing spaces in host & username text fields and trailing slashes for host as well

## What's new in version 2.0.2

### General improvements :

- Ask for confirmation before closing the app
- Auto detect white or black text color depending on text area background color
- Add recursive folder download feature

### Bug fixes :

- Fix thumbnail URL encoding issue on local site
- Tests against all supported file formats, bug fixing

## What's new in version 2.0.3

### New features :

- Bypass proxy

- Delete directories on server side (only if empty and not a root one)
- Rename directories on server side (only if empty and not a root one)
- Delete slides on server side
- Rename slides on server side
- Add copy full path option both locally and remotely

#### General improvements :

- Directly download to selected folder on local site
- Keyboard input interaction
- New icon for root directories
- Better transfer speed metrics
- Better error messages
- Enhanced integrity check
- Adding counters to all bottom bar buttons

#### Bug fixes :

- Refresh table view for local site following successful downloads

#### **What's new in version 2.1.0**

#### New features :

- Interfacing with the new My Pathomation API
- Adding create/rename/delete folder features for My Pathomation accounts
- Adding rename/delete slide features for My Pathomation accounts
- Adding drag & drop functionality for download
- Release of a self-containing package for Windows (No installation of JRE required)

#### General improvements :

- Detection of PMA.core version on connection
- Harmonization of upload/download UI/UX
- Cleaning up UI for slides without barcode
- Defining explicit location while adding new folders

#### Bug fixes :

- Disabling of upload/download buttons on disconnection

#### **What's new in version 2.1.1**

#### General improvements :

- Compliance with PMA.start and PMA.start dotnetcore on Windows
- Adding support of VPN connections

## What's new in version 2.2.0

### New features :

- Server to server transfers
- Multiple selection of slides for transfer

### General improvements :

- Compliance with PMA.start dotnetcore on MacOs & Unix
- Improved error messages (transfer of empty folders, transfers to location that require elevation of privileges..)
- Improved integrity check process

### Bug fixes :

- Better handling of failed connections
- Fixes on the site manager

## What's new in version 2.2.1

### New features :

- User quota enforcement

### General improvements :

- Error handling for uploads to Azure directories
- Improved error handling for server to server transfers
- Support for My Pathomation "add slide" API endpoint & "caller ID" parameter for slides uploaded via PMA.transfer
- Support for Keyboard key <Enter>

### Bug fixes :

- Fixes on the server to server progress bar

## What's new in version 2.2.2

### New features :

- Improved reporting of successful slide transfers (interactive vs. console)

### Bug fixes :

- Slide (file)size is reported correctly again
- Connectivity to My Pathomation is operational again

### What's new in version 2.2.3

#### New features :

- Fill folders with slides and subfolders recursively
- Unload multiple selected slides
- Delete several selected slides
- New folder in the cloud S3 directory

#### Bug fixes :

- Recursion for delete, download and upload

### What's new in version 2.2.4

#### General improvements :

- Upload and Download slides execute thru PMA-JAVA Maven dependency

#### New features :

- Renaming local folder
- Renaming local slide

#### Bug fixes :

- Progressive bar calculation in multi-component slides

### What's new in version 2.2.5

#### General improvements :

- Fixed compatibility issues with Pma.Core.

#### New features :

- Added support for uploading slides larger than 5 GB to Azure and Amazon.

From:  
<https://docs.pathomation.com/PMA.transfer/> - **PMA.transfer**

Permanent link:  
[https://docs.pathomation.com/PMA.transfer/doku.php?id=release\\_history&rev=1757661261](https://docs.pathomation.com/PMA.transfer/doku.php?id=release_history&rev=1757661261)

Last update: **2025/09/12 10:14**

