

What's new in version 2.2.3.d086727c81?

PMA.transfer 2.2.3.d086727c81 is an improved version of the previous PMA.transfer with fixed bugs and new features in functionality.

General improvements

- Upload slides and folders to the Amazon s3 cloud.
- Upload slides and folders recursively.
- Displaying information about slides in the cloud directories window.
- Adding a new folder in the cloud directory.
- Full functionality of work with My Pathomation cloud slides and directories(Upload, download, rename, delete)

New features in functionality

- Added the possibility to fill folders with slides and subfolders recursively, this means that you can load a folder containing subfolders containing both folders and slides.
- Added the possibility to unload multiple selected slides.
- Added the possibility to delete several selected slides from s3 the cloud directory at once.
- Added the possibility to create a new folder in the cloud S3 directory.
- Added display in the S3 cloud directory of slide data such as: Slidesize, FileType, LastModified.
- Added load single large format files to My Pathomation.
- The information window that the user always sees when a slide or folder is renamed or deleted from My Pathomation.
- In the right window items, right-click to open the slide in the browser - Core or My Pathomation.

Bug fixed

- Bug fixed preventing more than one slide from being uploaded.
- Bug fixed preventing more than one slide from being deleted.
- Bug fixed preventing more than one folder from being deleted.
- Bug fixed preventing making a new folder in the cloud directory.
- Bug fixed preventing load recursive folder with first empty folder and also last empty folder.
- Bug fixed in progress bar format count when single large slides downloads.

Slide formats

Slide formats with PMA.transfer has been tested:

```
* .mrXS
* .SVS
```

Last update:
2022/11/04 12:08

what_s_new_in_version_2.2.3 https://docs.pathomation.com/PMA.transfer/doku.php?id=what_s_new_in_version_2.2.3

- * .ndpi
- * .vsi
- * .jpg
- * .png
- * .zif
- * .tiff

From:

<https://docs.pathomation.com/PMA.transfer/> - **PMA.transfer**

Permanent link:

https://docs.pathomation.com/PMA.transfer/doku.php?id=what_s_new_in_version_2.2.3

Last update: **2022/11/04 12:08**

