

# What's new in version 2.2.3.d086727c81?

PMA.transfer 2.2.3.d086727c81 is an improved version of the previous PMA.transfer with fixed bugs and new features in functionality.

## General improvements

- Upload slides and folders to the Amazon s3 cloud.
- Upload slides and folders recursively.
- Displaying information about slides in the cloud directories window.
- Adding a new folder in the cloud directory.
- Full functionality of work with My Pathomation cloud slides and directories(Upload, download, rename, delete)

## New features in functionality

- Added the possibility to fill folders with slides and subfolders recursively, this means that you can load a folder containing subfolders containing both folders and slides.
- Added the possibility to unload multiple selected slides.
- Added the possibility to delete several selected slides from s3 the cloud directory at once.
- Added the possibility to create a new folder in the cloud S3 directory.
- Added display in the S3 cloud directory of slide data such as: Slidesize, FileType, LastModified.
- Added load single large format files to My Pathomation.
- The information window that the user always sees when a slide or folder is renamed or deleted from My Pathomation.
- In the right window items, right-click to open the slide in the browser - Core or in My Pathomation.

## Bug fixed

- Bug fixed preventing more than one slide from being uploaded.
- Bug fixed preventing more than one slide from being deleted.
- Bug fixed preventing more than one folder from being deleted.
- Bug fixed preventing making a new folder in the cloud directory.
- Bug fixed preventing load recursive folder with first empty folder and also last empty folder.
- Bug fixed in progress bar format count when single large slides downloads.

## Slide formats

**Slide formats with PMA.transfer has been tested:**

\* .mrxs

Last update: 2022/11/04 12:08 what\_s\_new\_in\_version\_2.2.3 [https://docs.pathomation.com/PMA.transfer/doku.php?id=what\\_s\\_new\\_in\\_version\\_2.2.3&rev=1667552919](https://docs.pathomation.com/PMA.transfer/doku.php?id=what_s_new_in_version_2.2.3&rev=1667552919)

---

- \* .svs
- \* .ndpi
- \* .vsi
- \* .jpg
- \* .png
- \* .zif
- \* .tiff

From: <https://docs.pathomation.com/PMA.transfer/> - **PMA.transfer**

Permanent link: [https://docs.pathomation.com/PMA.transfer/doku.php?id=what\\_s\\_new\\_in\\_version\\_2.2.3&rev=1667552919](https://docs.pathomation.com/PMA.transfer/doku.php?id=what_s_new_in_version_2.2.3&rev=1667552919)

Last update: **2022/11/04 12:08**

