


Accessing the AI image analysis menu

Start by selecting the slide you want to analyse. In the slide menu there is an AI button that opens the menu of available algorithms.

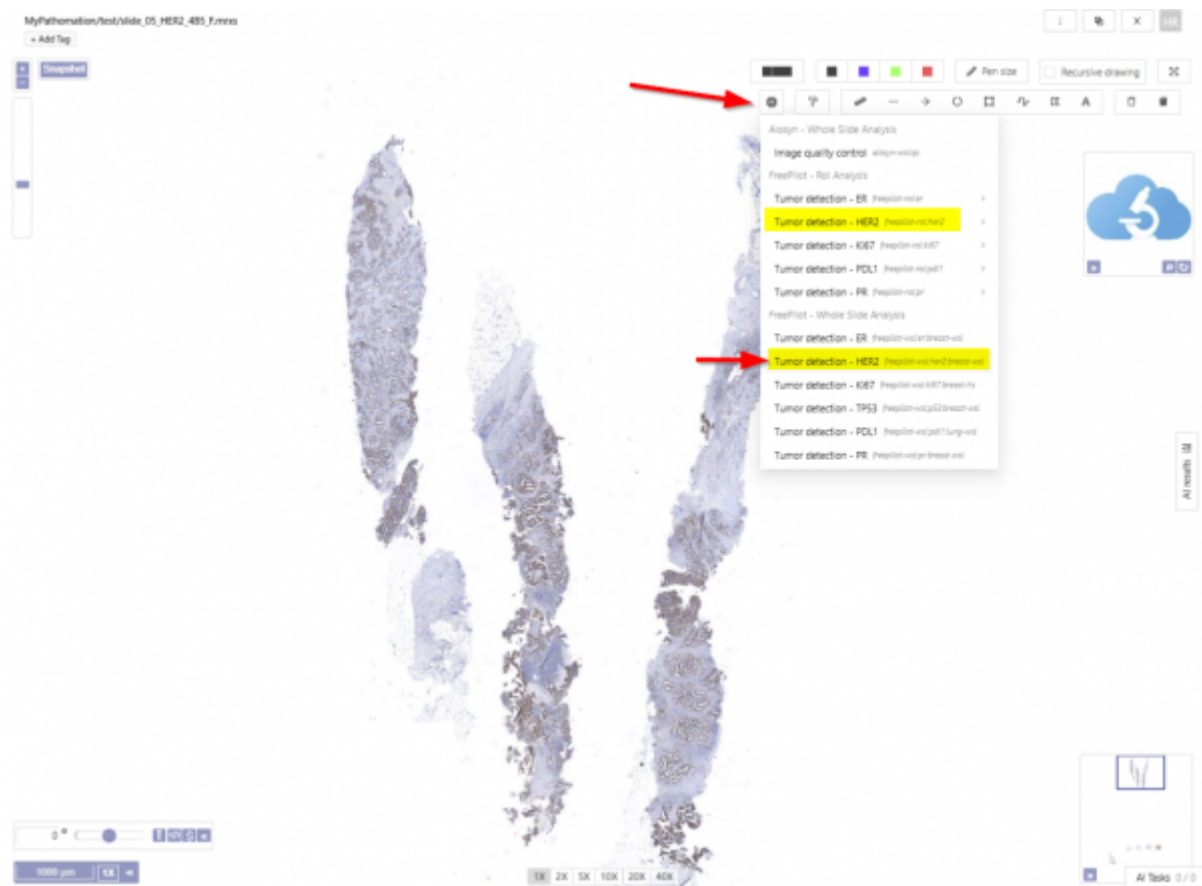

You can select an algorithm compatible with the slide and the intended use. The menu has a standard naming convention logic to make this clear: All algorithms are grouped per provider and then further subdivided if they are intended for the full slide WSI or for a Region of interest (ROI)

Vendor name - Whole slide analysis

- ***AI algorithm name*** - *vendor name wsi*

From: <https://docs.pathomation.com/my/> - **My Pathomation**

Permanent link: https://docs.pathomation.com/my/doku.php?id=accessing_the_ai_menu&rev=1752496948

Last update: **2025/07/14 15:42**

