


## Accessing the AI image analysis menu

Start by selecting the slide you want to analyse. In the slide menu there is an AI button that opens the menu of available algorithms.

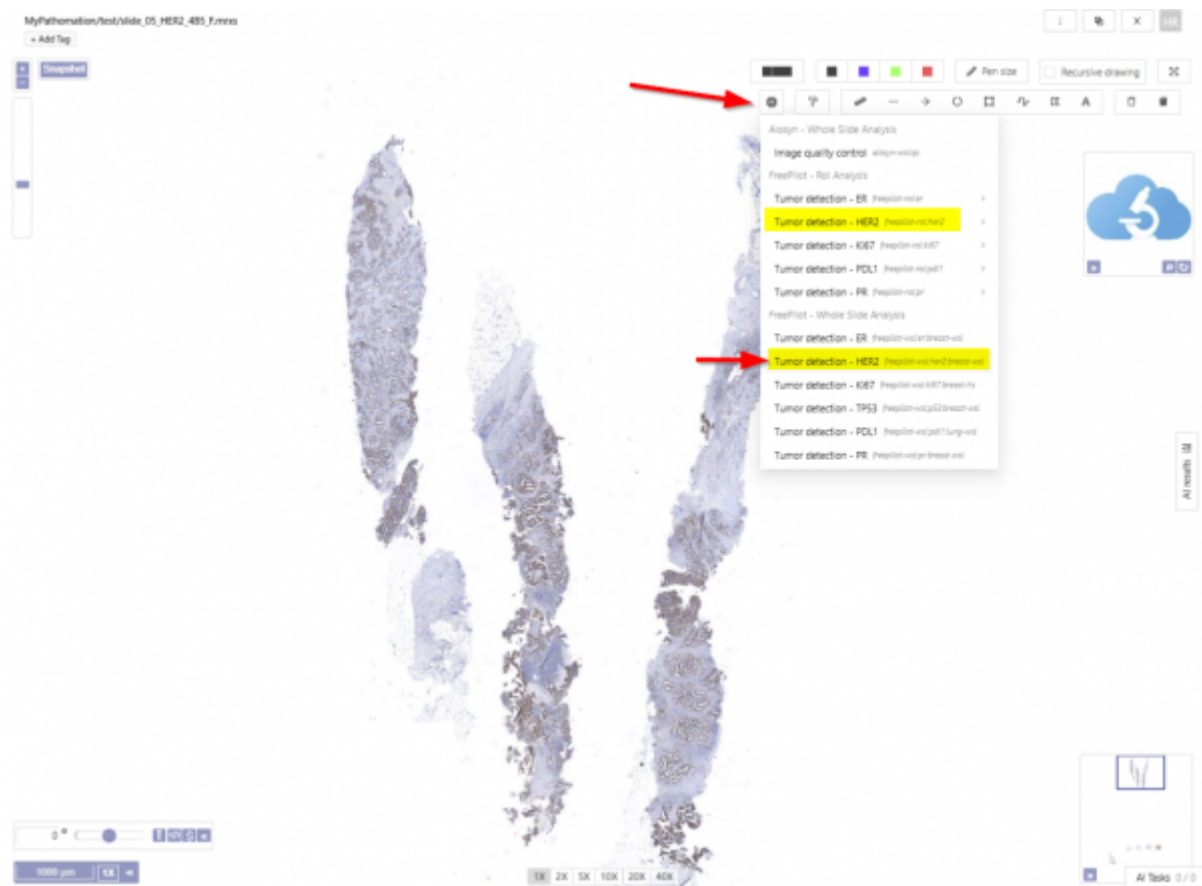


You can select an algorithm compatible with the slide and the intended use. The menu has a standard naming convention logic to make this clear: All algorithms are grouped per provider and then further subdivided if they are intended for the full slide WSI or for a Region of interest (ROI)


### *Vendor name - Whole slide analysis*

- *\*AI algorithm name\*\* - vendor name wsi*

Click on the algorithm you wish to use. For an ROI algorithm you have to either make an area annotation first and select it or opt for a field of view analysis.

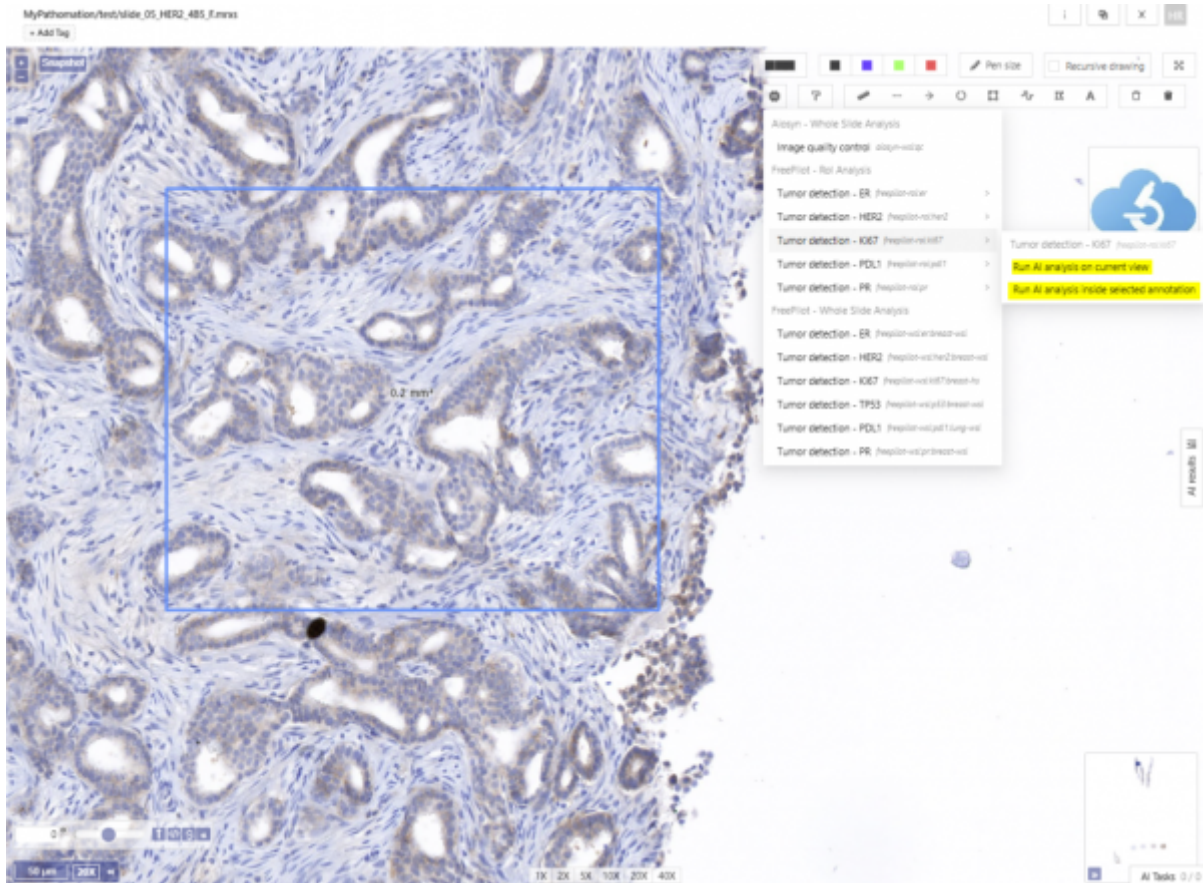

Once you select the AI algorithm of your choice a dialog window opens. You will see (1) the background information regarding the algorithm (2) the cost and balance of your account, if these are correct you can (3) run the analysis.

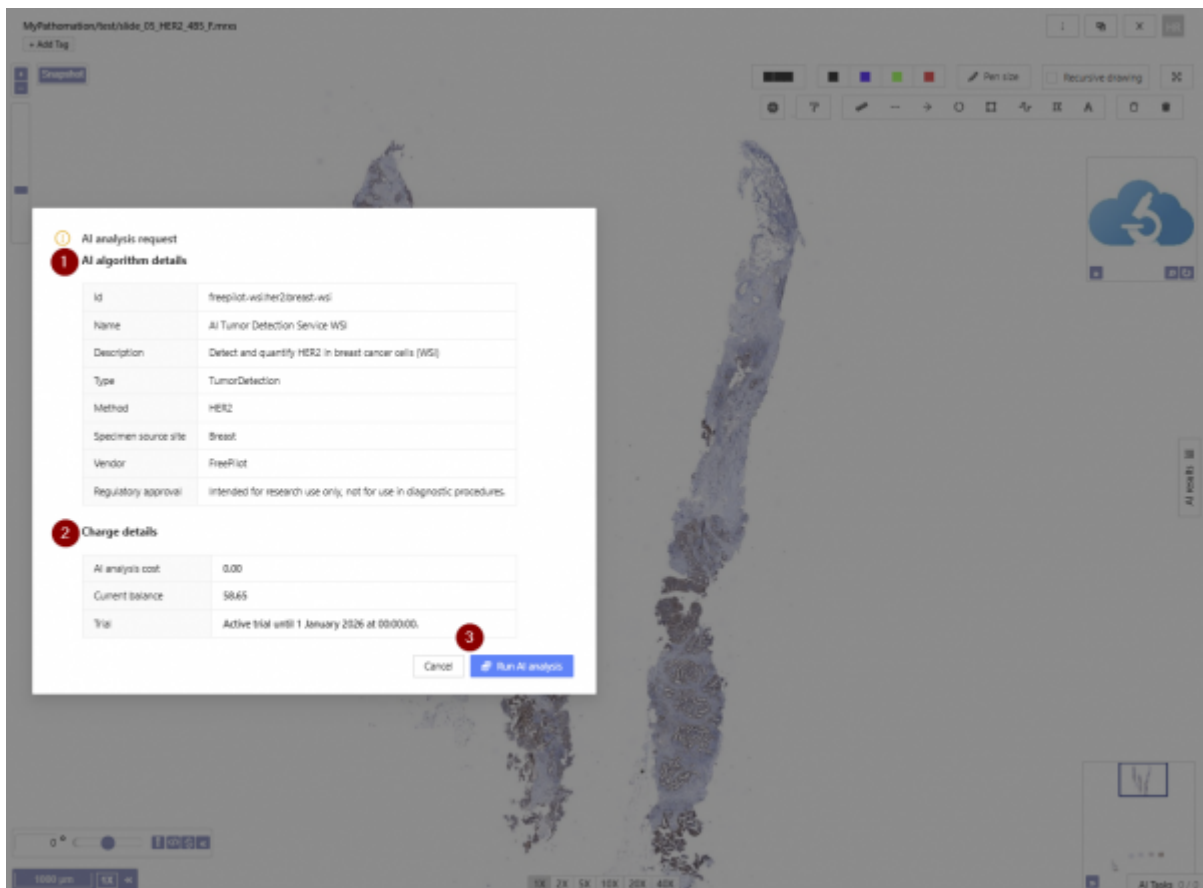

From:  
<https://docs.pathomation.com/my/> - **My Pathomation**

Permanent link:  
[https://docs.pathomation.com/my/doku.php?id=accessing\\_the\\_ai\\_menu&rev=1752497880](https://docs.pathomation.com/my/doku.php?id=accessing_the_ai_menu&rev=1752497880)

Last update: **2025/07/14 15:58**

