


Introduction

My Pathomation provides you with an easy way to create a dedicated case sharing space per Pathology speciality and manage the persons with access to review these cases

This is based on the collection feature in combination with the groups feature.

For each pathology speciality for which you need expert reviewers, create a 'Sharing and review' collection and then create a group to link it with the other My Pathomation users in your organization that will need to review the cases.

The example below shows the workflow, first you assign full cases or selected images of a case from your uploads to a collection. Every time a new case is added to the 'Sharing and review' collection the users will be notified there is a case to review.

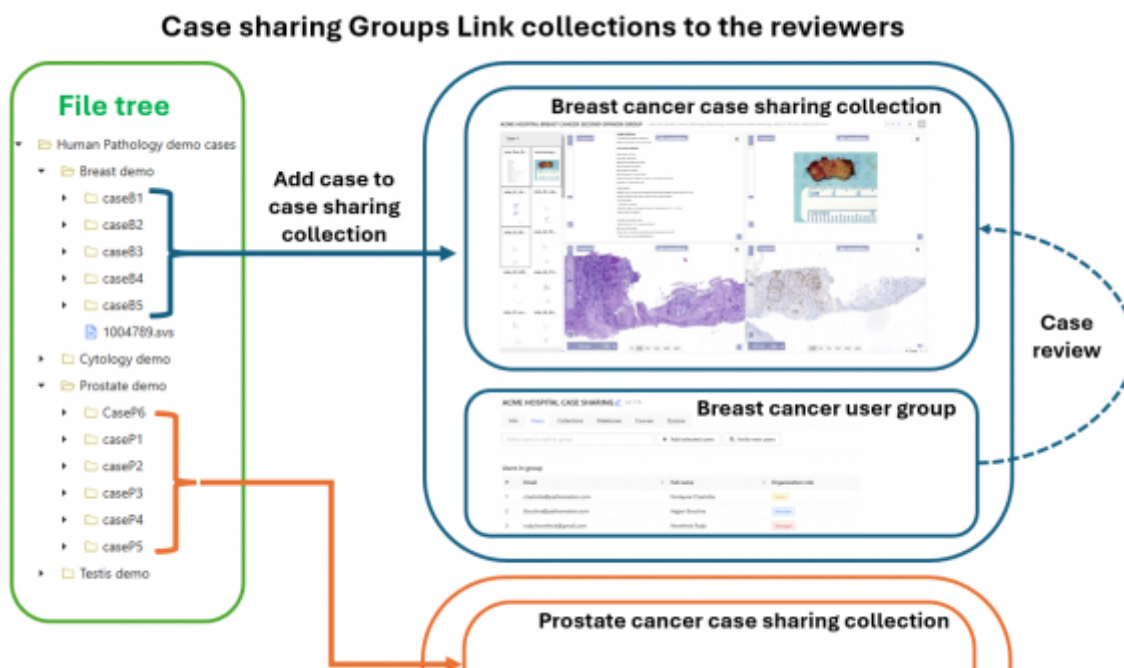

Your workflow can also be set up per hospital pathology lab you collaborate with in a similar fashion..

From: <https://docs.pathomation.com/my/> - My Pathomation

Permanent link: https://docs.pathomation.com/my/doku.php?id=case_sharing_workflow_introduction&rev=1753698521

Last update: 2025/07/28 13:28

