



How to add a collection?

1) Go to “My collections” in the sidebar.

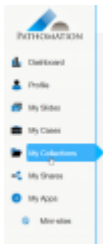

2) Click on “Add collection” on the top right of the page.

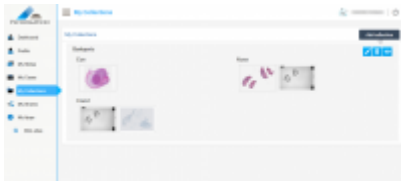

3) Choose a name for your collection, add a logo if necessary and click on “Create”.



4) Select the cases which you want to add to your collection and press “Add selected”.



5) Return to the list, by using the “Back to list” button.



From: <https://docs.pathomation.com/my/> - My Pathomation

Permanent link: https://docs.pathomation.com/my/doku.php?id=how_to_add_a_collection&rev=1580811583

Last update: 2020/02/04 13:19

