



## How to add collections?

1) Go to “My collections” in the sidebar.

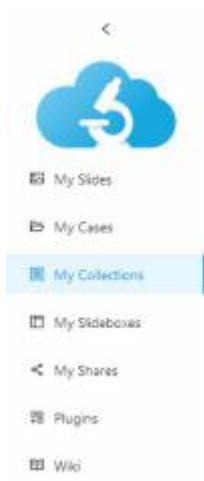


2) Click on “+ New collection” on the top right of the page.

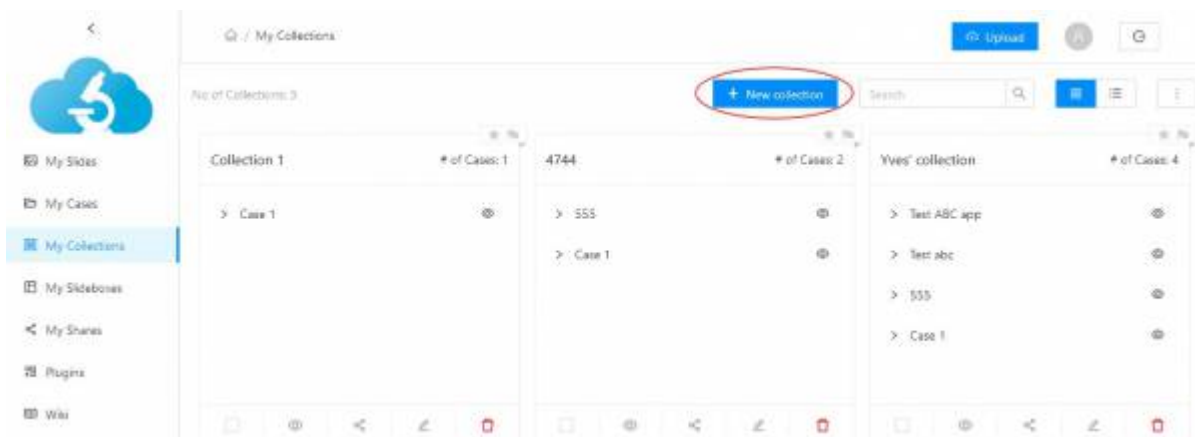


3) Choose a name for your collection and click on “Create”.



4) Select the cases which you want to add to your collection and press “>”.

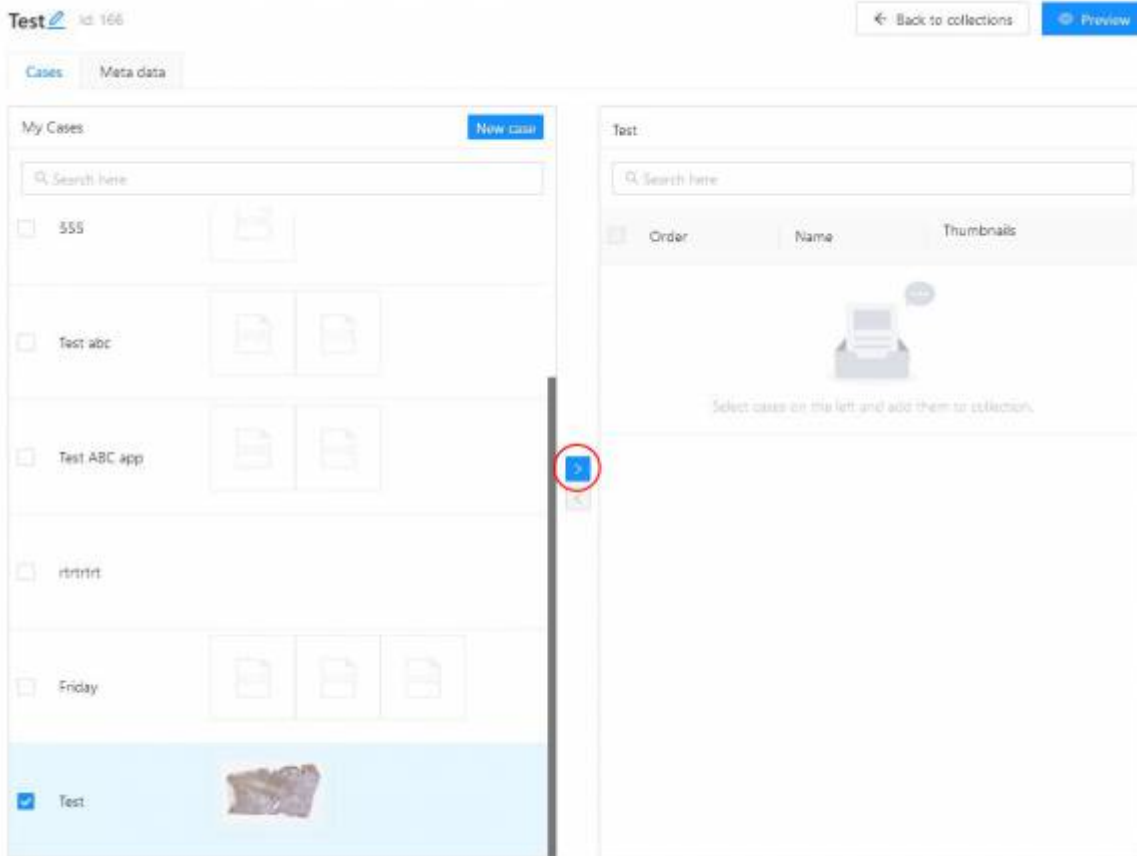

5) Return to the list, by using the “Back to collections” button, or preview your newly created collection by clicking on “Preview” button.

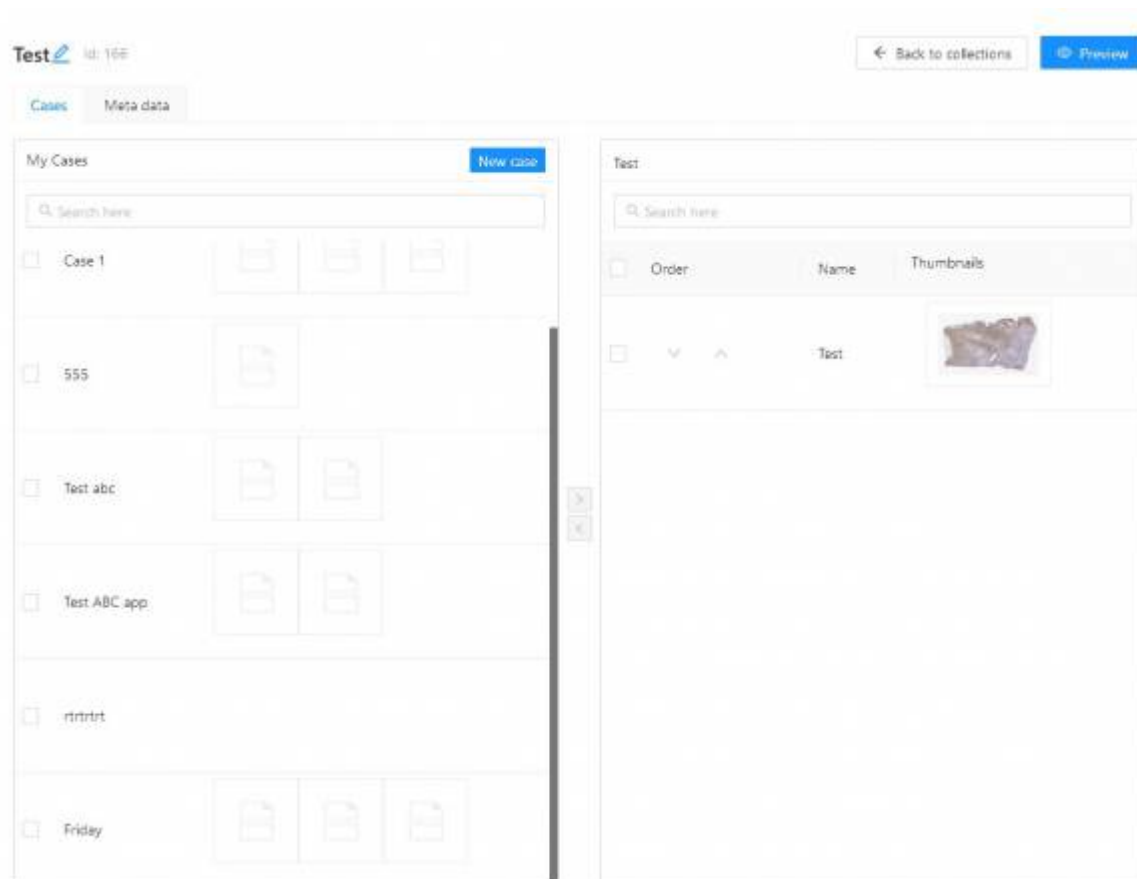

From:  
<https://docs.pathomation.com/my/> - **My Pathomation**

Permanent link:  
[https://docs.pathomation.com/my/doku.php?id=how\\_to\\_add\\_a\\_collection&rev=1644831075](https://docs.pathomation.com/my/doku.php?id=how_to_add_a_collection&rev=1644831075)

Last update: **2022/02/14 12:31**

