



### How to add collections?

1) Go to “My collections” in the sidebar.

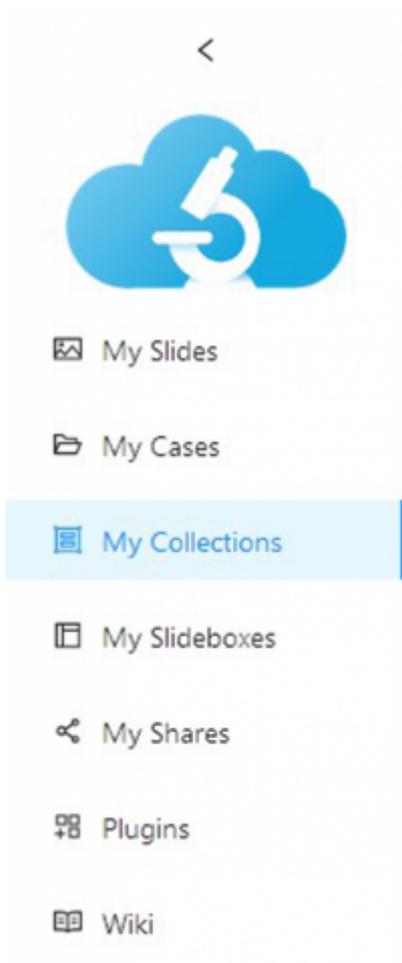

2) Click on “+ New collection” on the top right of the page.

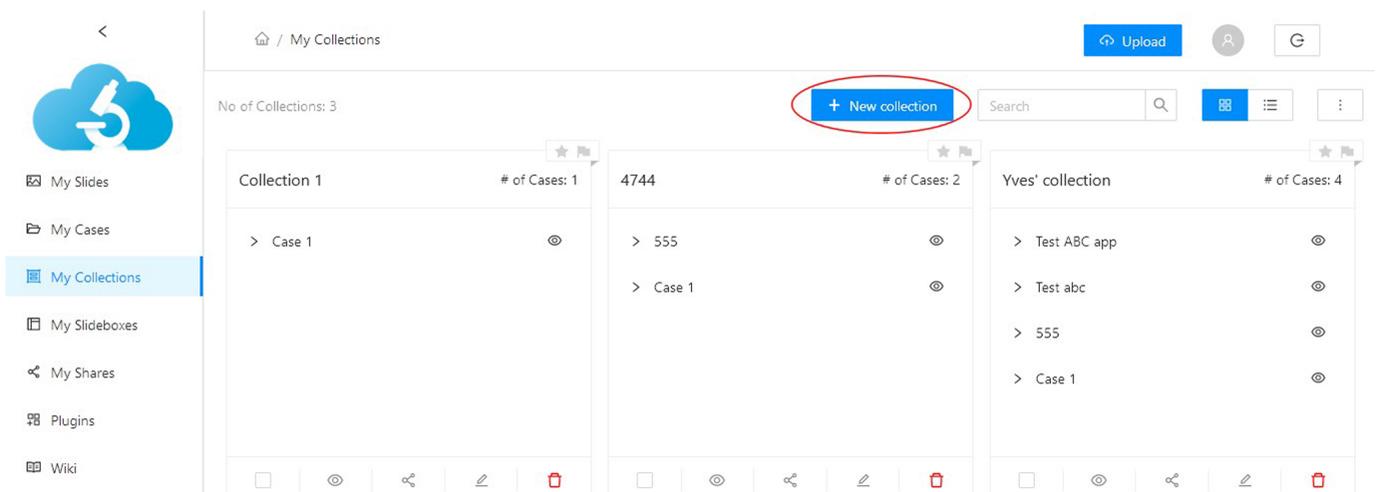

3) Choose a name for your collection and click on “Create”.

## New collection



\* Name of new collection

New collection name

Cancel

Create

4) Select the cases which you want to add to your collection and press ">".

Test Id: 166

[← Back to collections](#)

[Preview](#)

Cases Meta data

The screenshot shows the 'Test' collection interface. On the left, under 'My Cases', there is a list of cases with checkboxes and thumbnails. The 'Test' case is selected. On the right, the 'Test' collection preview area is shown with a search bar and a table header. A red circle highlights the right arrow button between the two panels.

Order	Name	Thumbnails
<input type="checkbox"/>	555	
<input type="checkbox"/>	Test abc	
<input type="checkbox"/>	Test ABC app	
<input type="checkbox"/>	rttrtrtrt	
<input type="checkbox"/>	Friday	
<input checked="" type="checkbox"/>	Test	

5) Return to the list, by using the "Back to collections" button, or preview your newly created collection by clicking on "Preview" button.

Test  Id: 166

[← Back to collections](#)

[Preview](#)

[Cases](#) [Meta data](#)

My Cases [New case](#)

Search here

<input type="checkbox"/>	Case 1			
<input type="checkbox"/>	555			
<input type="checkbox"/>	Test abc			
<input type="checkbox"/>	Test ABC app			
<input type="checkbox"/>	rtrtrtrt			
<input type="checkbox"/>	Friday			

Test

Search here

<input type="checkbox"/>	Order	Name	Thumbnails
<input type="checkbox"/>	▼ ▲	Test	

From: <https://docs.pathomation.com/my/> - My Pathomation

Permanent link: [https://docs.pathomation.com/my/doku.php?id=how\\_to\\_add\\_a\\_collection&rev=1644842283](https://docs.pathomation.com/my/doku.php?id=how_to_add_a_collection&rev=1644842283)

Last update: **2022/02/14 15:38**

