


How to add collections?

1) Go to “My collections” in the sidebar.

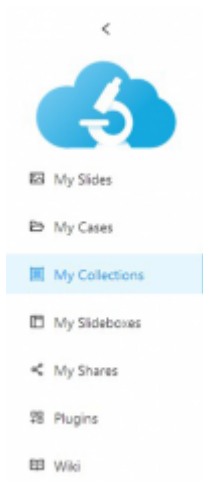


2) Click on “+ New collection” in the middle of the page.

3) Choose a name for your collection and click on “Create”.

4) Now you can choose to add single slides to your collection or to create a case within your collection.

5) you can now preview your newly created collection by clicking on “Preview” button.

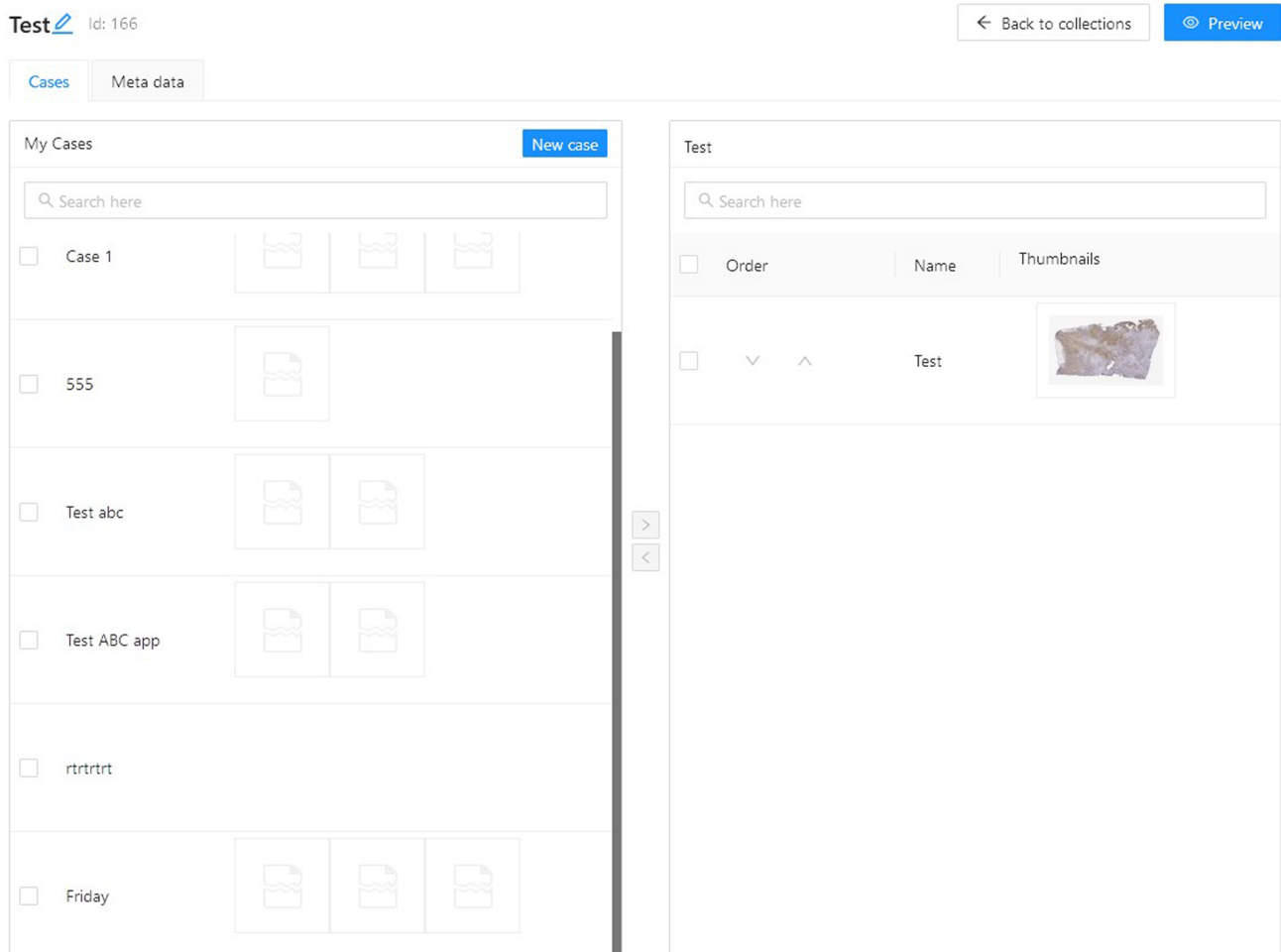

From:
<https://docs.pathomation.com/my/> - **My Pathomation**

Permanent link:
https://docs.pathomation.com/my/doku.php?id=how_to_add_a_collection&rev=1662381300

Last update: **2022/09/05 15:35**

