



### How to add collections?


1) Go to “My collections” in the sidebar.

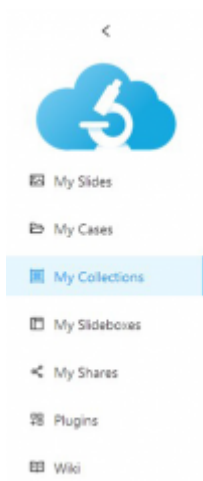


2) Click on “+ New collection” in the middle of the page.

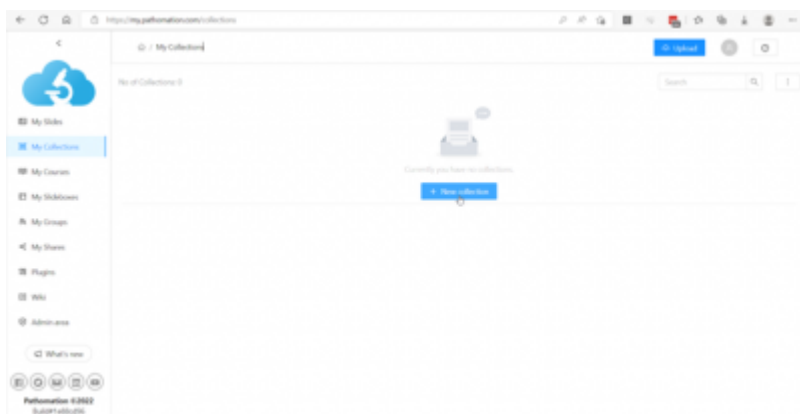


3) Choose a name for your collection and click on “Create”.

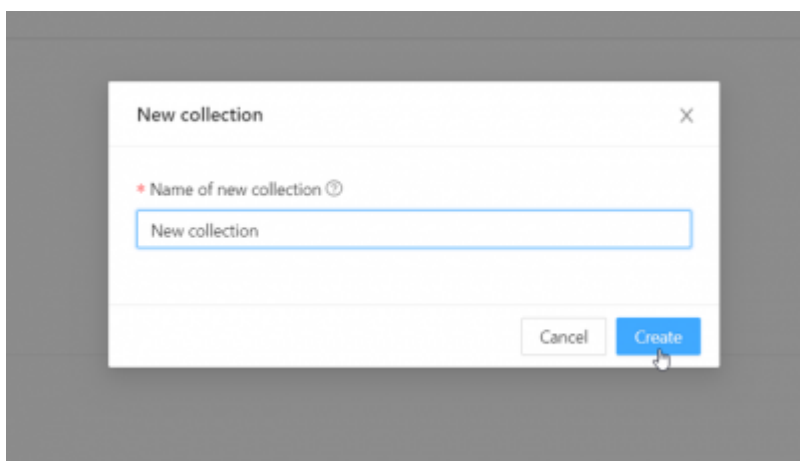


4) Now you can choose to add single slides to your collection or to create a case within your collection.

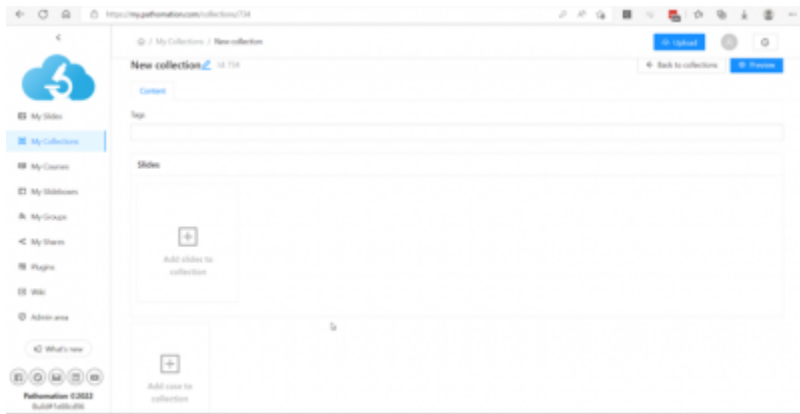



- To add a slide: you click on the “add slides” button and you choose a slide form your My Pathomation slide repository

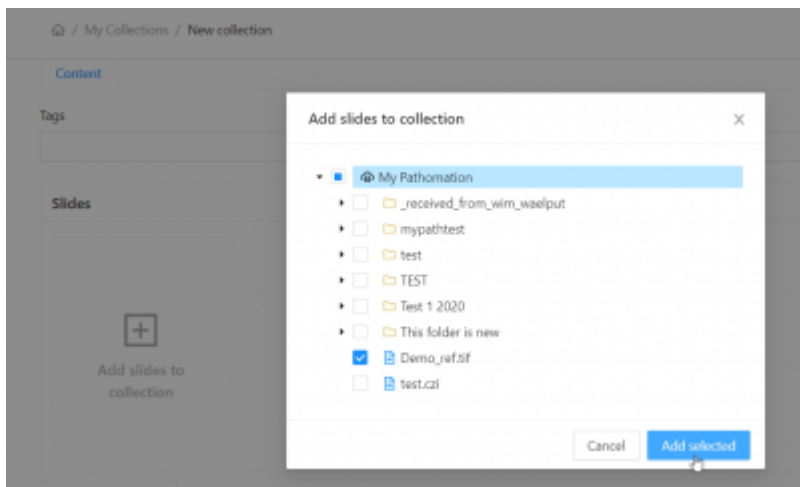

- To add a case: you click on the “add case” button. First you will need to create a name for the case you would like to create.

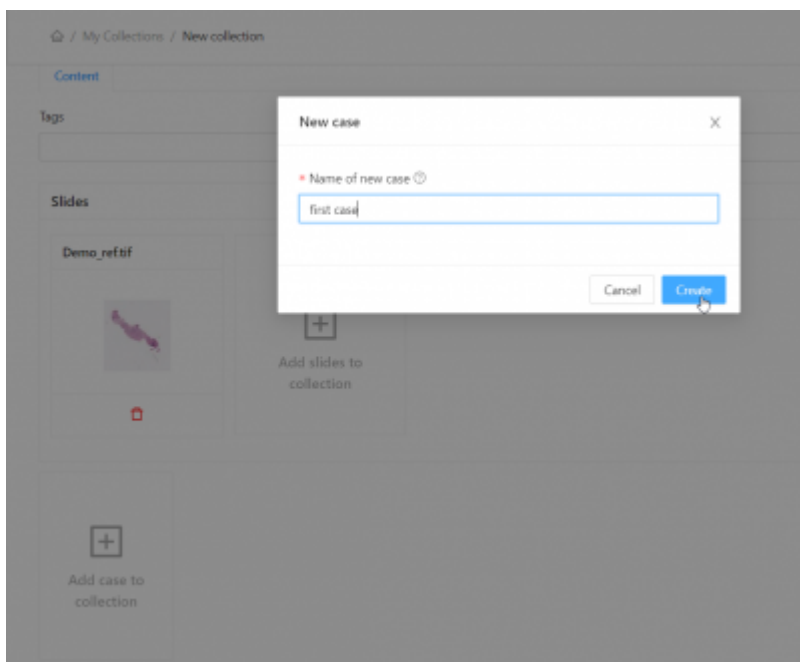


Now you are able to add slides to your case

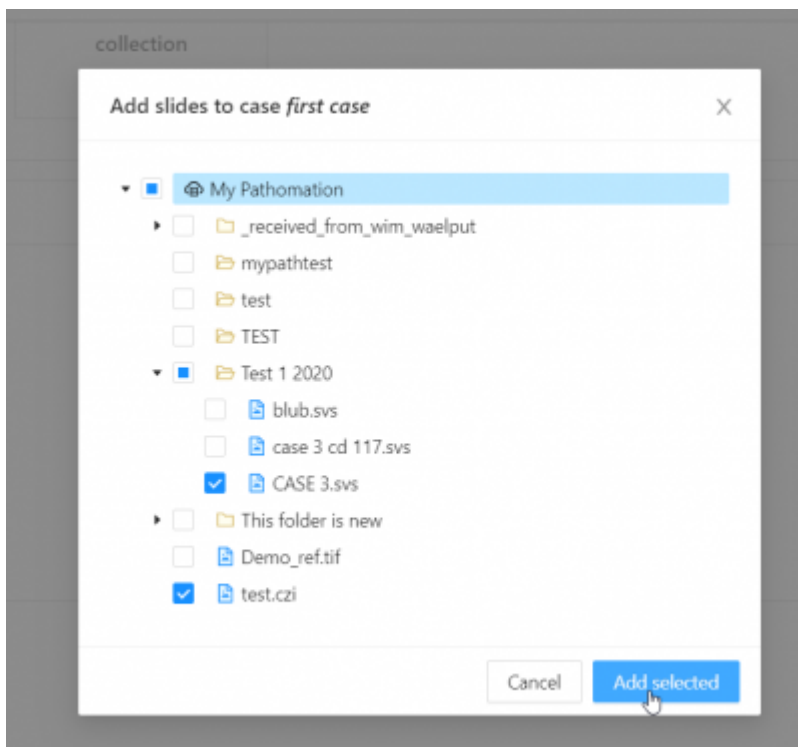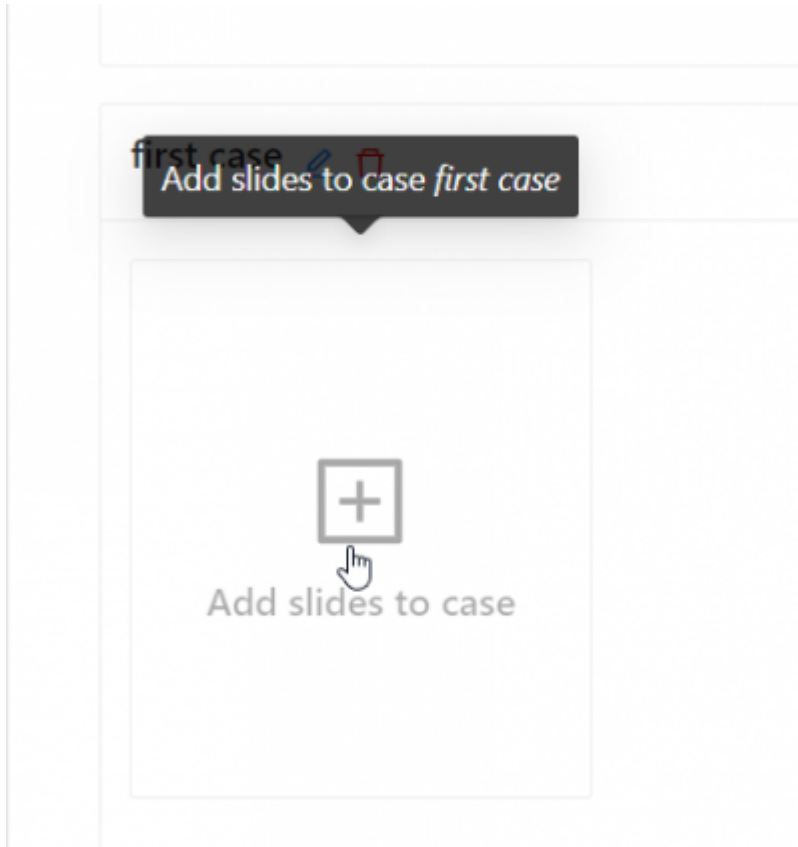

5) you can now preview your newly created collection by clicking on "Preview" button.

Last update:  
2022/09/05 16:11

how\_to\_add\_a\_collection [https://docs.pathomation.com/my/doku.php?id=how\\_to\\_add\\_a\\_collection&rev=1662383519](https://docs.pathomation.com/my/doku.php?id=how_to_add_a_collection&rev=1662383519)

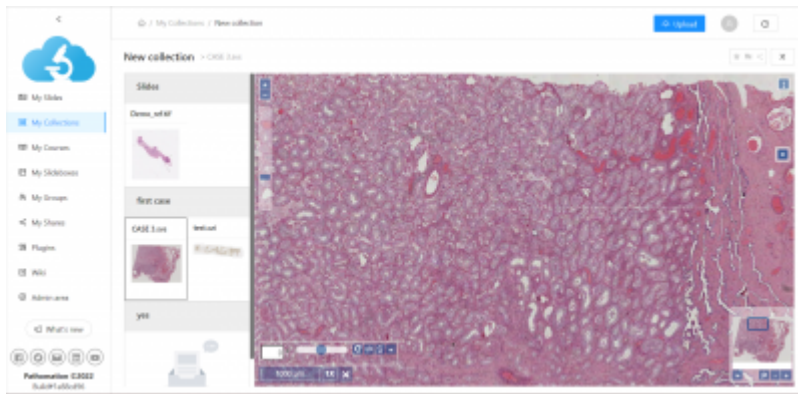

From:  
<https://docs.pathomation.com/my/> - My Pathomation

Permanent link:  
[https://docs.pathomation.com/my/doku.php?id=how\\_to\\_add\\_a\\_collection&rev=1662383519](https://docs.pathomation.com/my/doku.php?id=how_to_add_a_collection&rev=1662383519)

Last update: **2022/09/05 16:11**

