


# How to add a slidebox?

Before you make a slidebox, make sure you've already uploaded some slides and created at least one collection to add to your slidebox.


1. Go to "slidebox" in the sidebar

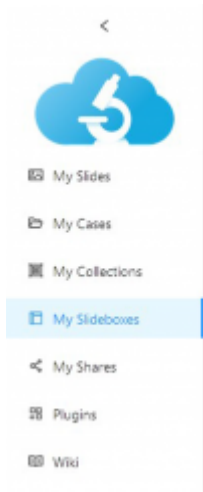


2. Click on "+ New slidebox" on the top right of the page




3. Choose a name for your slidebox and click on "Create".

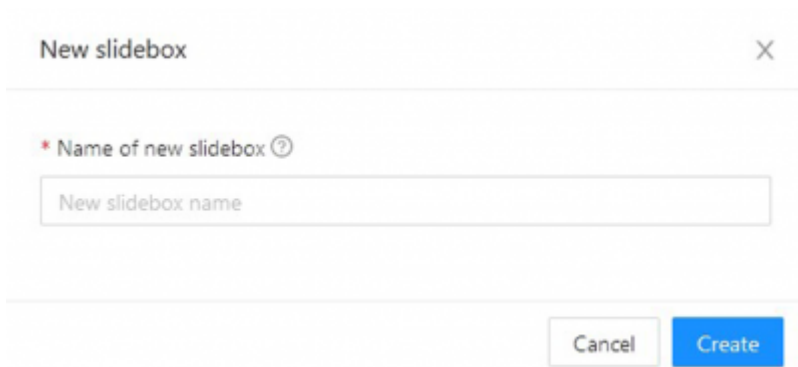

4) Select the collections which you want to add to your slidebox and press ">".

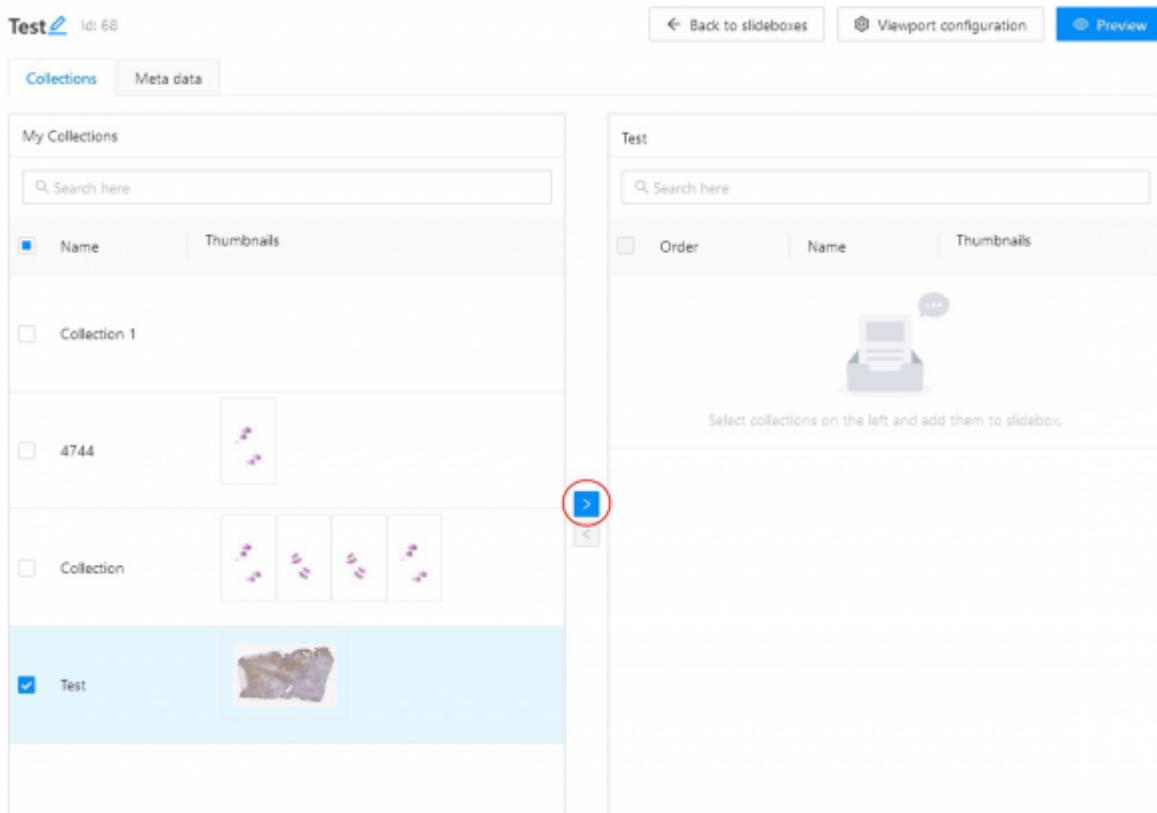

5) Return to the list, by using the “Back to slideboxes” button, or preview your newly created slidebox by clicking on “Preview” button.

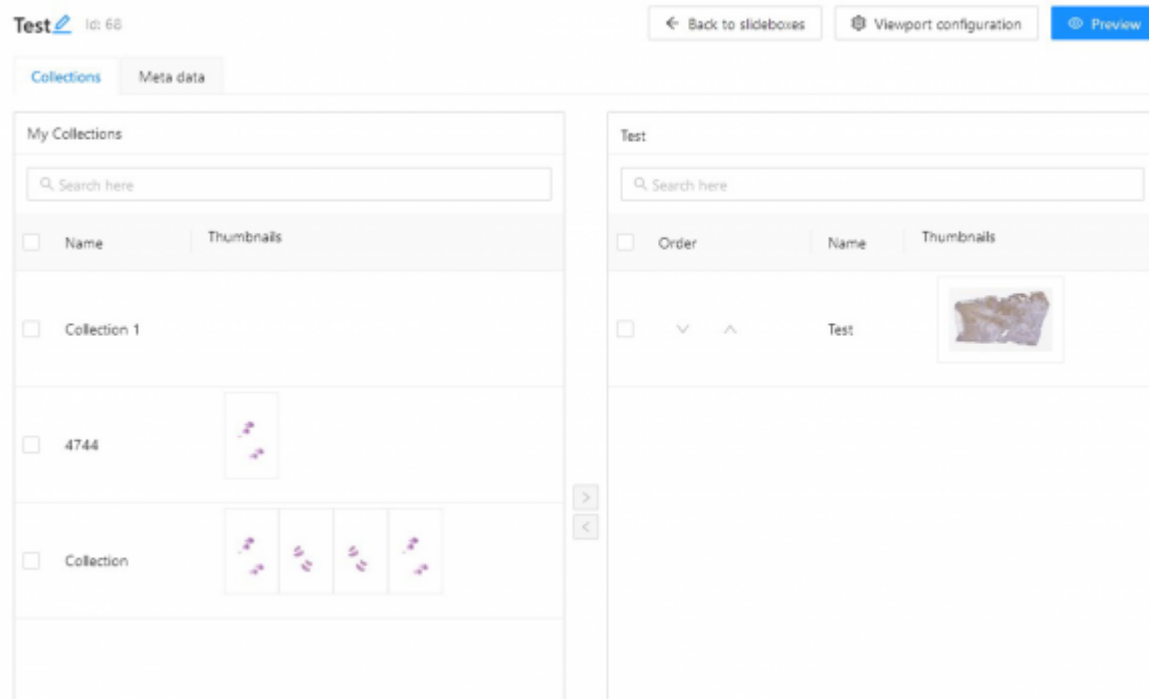

From:

<https://docs.pathomation.com/my/> - **My Pathomation**

Permanent link:

[https://docs.pathomation.com/my/doku.php?id=how\\_to\\_add\\_a\\_slidebox&rev=1644842156](https://docs.pathomation.com/my/doku.php?id=how_to_add_a_slidebox&rev=1644842156)

Last update: **2022/02/14 15:35**

