


# How to add a slidebox?

Before you make a slidebox, make sure you've already uploaded some slides and created at least one collection to add to your slidebox.

1. Go to "slidebox" in the sidebar

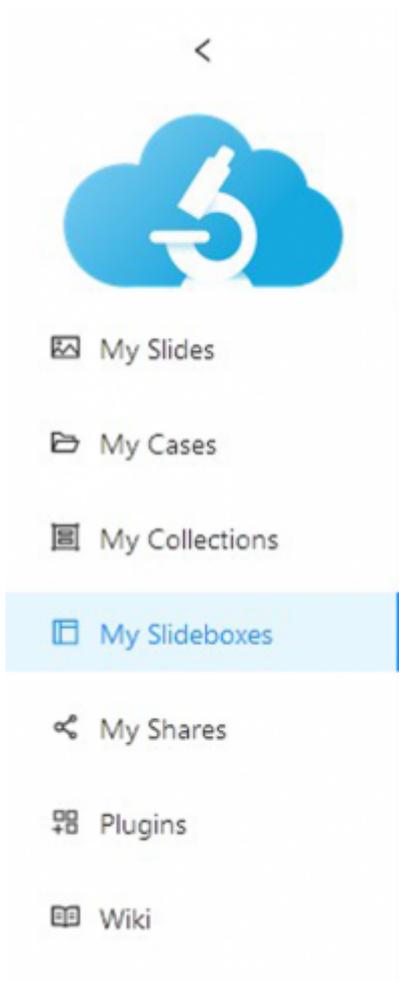

2. Click on "+ New slidebox" on the top right of the page



3. Choose a name for your slidebox and click on "Create".

## New slidebox



\* Name of new slidebox

Cancel

Create

4) Select the collections which you want to add to your slidebox and press ">".

Test Id: 68 ← Back to slideboxes ⚙ Viewport configuration 👁 Preview

**Collections** | Meta data

My Collections

🔍 Search here

| <input checked="" type="checkbox"/> | Name         | Thumbnails |
|-------------------------------------|--------------|------------|
| <input type="checkbox"/>            | Collection 1 |            |
| <input type="checkbox"/>            | 4744         |            |
| <input type="checkbox"/>            | Collection   |            |
| <input checked="" type="checkbox"/> | Test         |            |

Test

🔍 Search here

| <input type="checkbox"/>                                     | Order | Name | Thumbnails |
|--|-------|------|------------|
| <br>Select collections on the left and add them to slidebox. |       |      |            |

5) Return to the list, by using the "Back to slideboxes" button, or preview your newly created slidebox by clicking on "Preview" button.

Test  Id: 68

[← Back to slideboxes](#)

[⚙ Viewport configuration](#)

[👁 Preview](#)

[Collections](#) [Meta data](#)

| My Collections                        |   |
|---------------------------------------|---|
| 🔍 Search here                         |   |
| <input type="checkbox"/> Name         | Thumbnails  |
| <input type="checkbox"/> Collection 1 |   |
| <input type="checkbox"/> 4744         |  |
| <input type="checkbox"/> Collection   |  |

| Test                           |      |   |
|--------------------------------|------|---|
| 🔍 Search here                  |      |   |
| <input type="checkbox"/> Order | Name | Thumbnails  |
| <input type="checkbox"/> ▼ ^   | Test |  |

From: <https://docs.pathomation.com/my/> - **My Pathomation**

Permanent link: [https://docs.pathomation.com/my/doku.php?id=how\\_to\\_add\\_a\\_slidebox&rev=1644842337](https://docs.pathomation.com/my/doku.php?id=how_to_add_a_slidebox&rev=1644842337)

Last update: **2022/02/14 15:38**

