



How to add a slidebox?

Before you make a slidebox, make sure you've already uploaded some slides and created at least one collection to add to your slidebox.

1. Go to "slidebox" in the sidebar

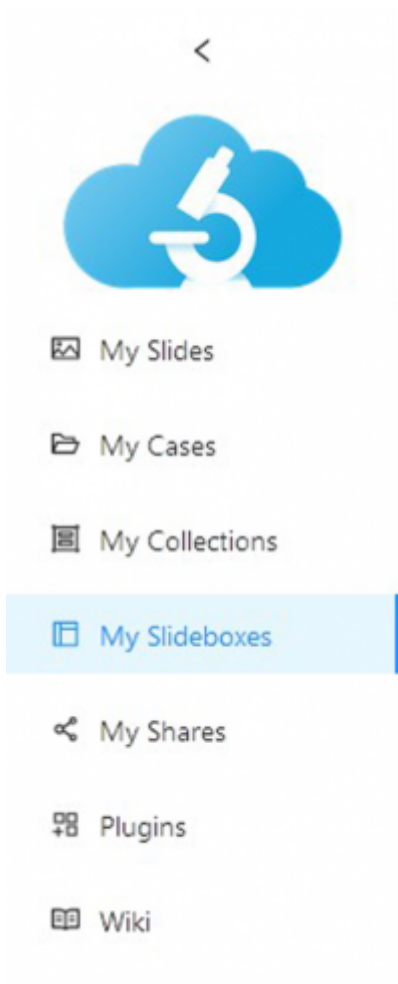

2. Click on "+ New slidebox" on the top right of the page

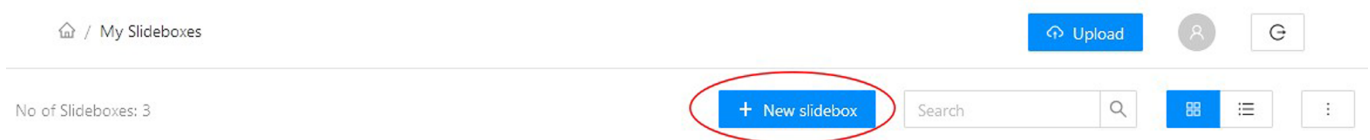

3. Choose a name for your slidebox and click on "Create".

New slidebox



* Name of new slidebox

Cancel

Create

4) Select the collections which you want to add to your slidebox and press ">".

Test Id: 68 ← Back to slideboxes ⚙ Viewport configuration 👁 Preview

Collections | Meta data

My Collections

🔍 Search here

<input checked="" type="checkbox"/>	Name	Thumbnails
<input type="checkbox"/>	Collection 1	
<input type="checkbox"/>	4744	
<input type="checkbox"/>	Collection	
<input checked="" type="checkbox"/>	Test	

Test

🔍 Search here

<input type="checkbox"/>	Order	Name	Thumbnails
 Select collections on the left and add them to slidebox.			

5) Return to the list, by using the "Back to slideboxes" button, or preview your newly created slidebox by clicking on "Preview" button.



Test  Id: 68


[← Back to slideboxes](#)

[⚙ Viewport configuration](#)

[👁 Preview](#)

[Collections](#) [Meta data](#)

My Collections	
🔍 Search here	
<input type="checkbox"/> Name	Thumbnails
<input type="checkbox"/> Collection 1	
<input type="checkbox"/> 4744	
<input type="checkbox"/> Collection	

Test		
🔍 Search here		
<input type="checkbox"/> Order	Name	Thumbnails
<input type="checkbox"/> ▼ ▲	Test	

From: <https://docs.pathomation.com/my/> - **My Pathomation**

Permanent link: https://docs.pathomation.com/my/doku.php?id=how_to_add_a_slidebox&rev=1644842337

Last update: **2022/02/14 15:38**

