



How to add a slidebox?

Before you make a slidebox, make sure you've already uploaded some slides and created at least one collection to add to your slidebox.


1. Go to "slidebox" in the sidebar

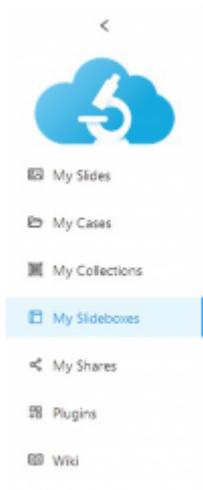

2. Click on "+ New slidebox" on the top right of the page

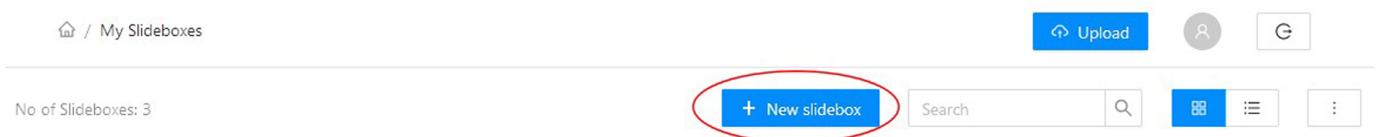

3. Choose a name for your slidebox and click on "Create".

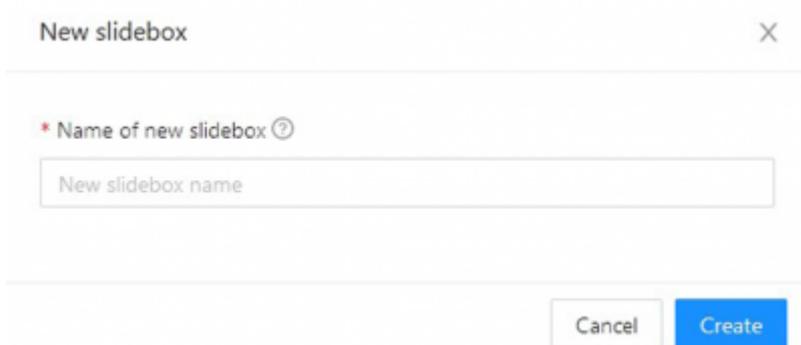

4) Select the collections which you want to add to your slidebox and press ">".

Test Id: 68 ← Back to slideboxes ⚙ Viewport configuration 👁 Preview

Collections | Meta data

My Collections

🔍 Search here

<input checked="" type="checkbox"/>	Name	Thumbnails
<input type="checkbox"/>	Collection 1	
<input type="checkbox"/>	4744	
<input type="checkbox"/>	Collection	
<input checked="" type="checkbox"/>	Test	

Test

🔍 Search here

<input type="checkbox"/>	Order	Name	Thumbnails
 Select collections on the left and add them to slidebox.			

▶ <

5) Return to the list, by using the “Back to slideboxes” button, or preview your newly created slidebox by clicking on “Preview” button.

Test Id: 68 ← Back to slideboxes ⚙ Viewport configuration 👁 Preview

Collections | Meta data

My Collections

🔍 Search here

<input type="checkbox"/>	Name	Thumbnails
<input type="checkbox"/>	Collection 1	
<input type="checkbox"/>	4744	
<input type="checkbox"/>	Collection	

Test

🔍 Search here

<input type="checkbox"/>	Order	Name	Thumbnails
<input type="checkbox"/>	▼ ^	Test	

> <

From:
<https://docs.pathomation.com/my/> - **My Pathomation**

Permanent link:
https://docs.pathomation.com/my/doku.php?id=how_to_add_a_slidebox&rev=1644842708

Last update: **2022/02/14 15:45**

