


How to Add a Slidebox

1. When to Use a Slidebox


Once you've uploaded some slides and created at least one collection, you can create a slidebox. A slidebox allows you to group and organize collections. Here you can also metadata to your collections. Some use cases for slideboxes include: Preparing your students for an end of year exam by grouping all teaching collections in one slidebox for studying, organising case collections from multiple hospitals in one place for pathologists with multiple residencies; or organising multiple timepoints in a longitudinal study.

2. Walkthrough


1. Go to "Slidebox" in the sidebar

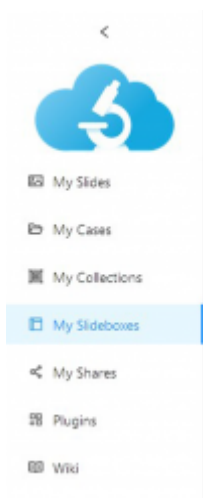


2. Click on "+ New slidebox" in the middle of the page

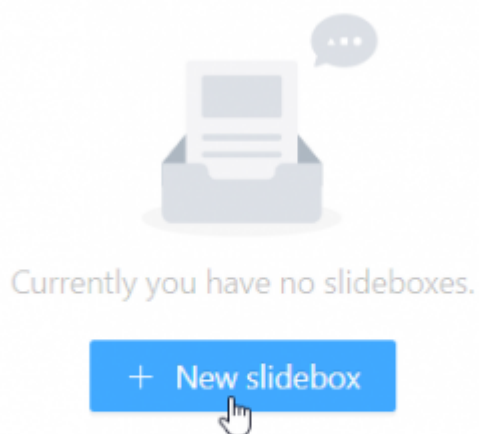

3. Choose a name for your slidebox and click on "Create".

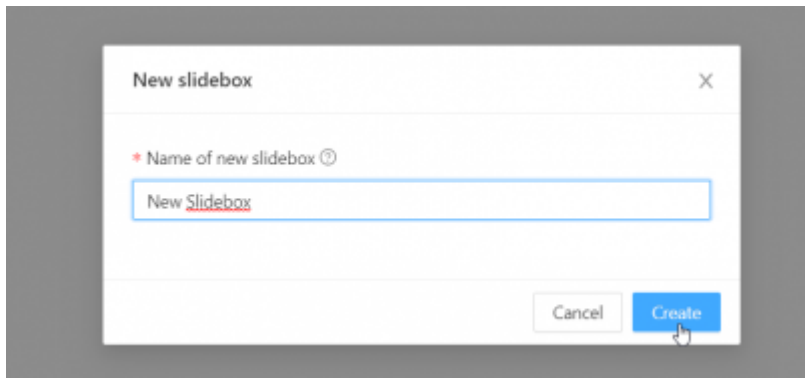


4) Select the content which you want to add to your slidebox and press “>”.

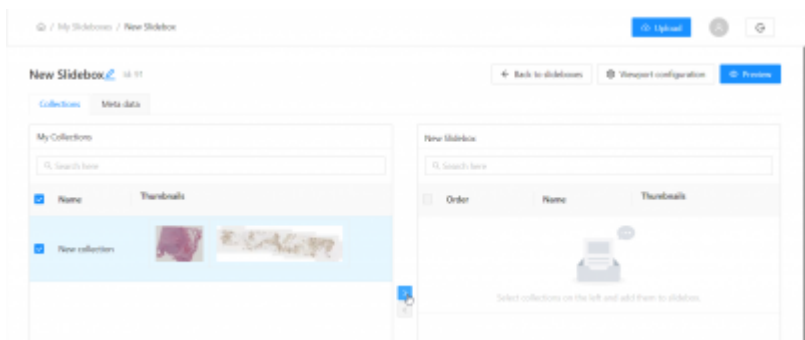

- You can also add meta-data to your slidebox



5) Also for Slideboxes in My Pathomation, you can configure the viewport and decide what people can see and do. Click on “viewport configuration” to choose how people can look at your slides.

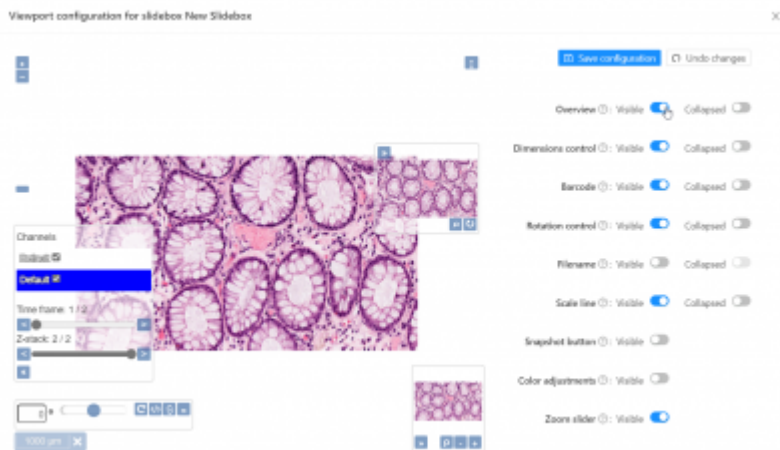

6) Return to the list, by using the “Back to slideboxes” button, or preview your newly created slidebox by clicking on “Preview” button.



Test  Id: 68


[← Back to slideboxes](#)

[⚙ Viewport configuration](#)

[👁 Preview](#)

[Collections](#) [Meta data](#)

My Collections	
<input type="text" value="Search here"/>	
<input type="checkbox"/> Name	Thumbnails
<input type="checkbox"/> Collection 1	
<input type="checkbox"/> 4744	
<input type="checkbox"/> Collection	

Test		
<input type="text" value="Search here"/>		
<input type="checkbox"/> Order	Name	Thumbnails
<input type="checkbox"/> ▼ ▲	Test	

From: <https://docs.pathomation.com/my/> - **My Pathomation**

Permanent link: https://docs.pathomation.com/my/doku.php?id=how_to_add_a_slidebox&rev=1752477506

Last update: **2025/07/14 10:18**

