


# How to add cases?

A case can only be created from within a collection.

1) Click on “+ New case” on the top right of the page.

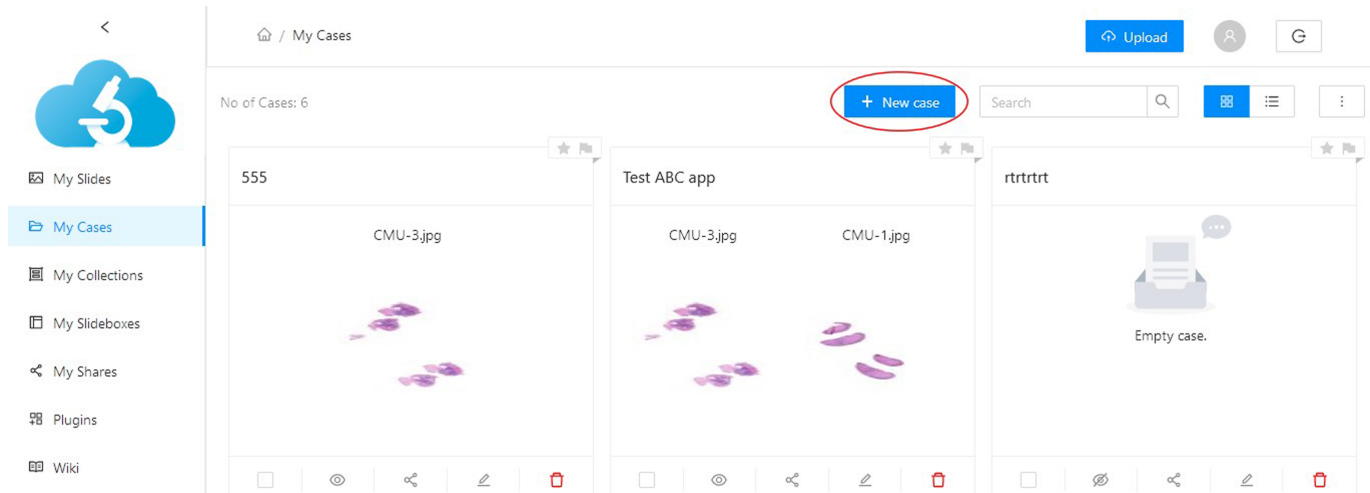

2) Choose a name for your case and press “Create”

A screenshot of the 'New case' dialog box. It has a title bar with 'New case' and a close button (X). Inside, there is a label '\* Name of new case ?' followed by a text input field containing the placeholder text 'New case name'. At the bottom right, there are two buttons: 'Cancel' and 'Create'.

3) Select the slides which you want to add to your case.

4) Click on “>” button

The slides now have been added to your case. To view your cases, go back to the list by using the “Back to cases” button, or preview your newly created case by clicking on “Preview” button.

From: <https://docs.pathomation.com/my/> - **My Pathomation**

<https://docs.pathomation.com/my/doku.php?id=how to add cases>

