


Adding a new slide can be easily done via the upload container (at the page bottom) :

To launch a new upload, you can either :

1. Click on the upload container, a file chooser window pops up, once you select the slide to upload, the upload is started automatically
2. Drag the slide to upload then drop it on the upload container, the upload then starts automatically

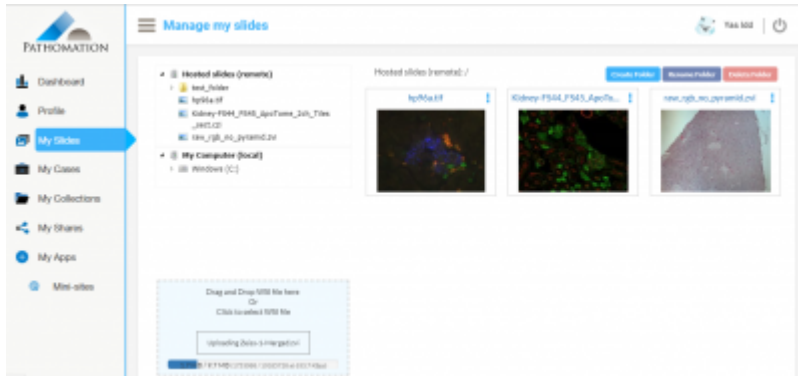

Once the slide is successfully uploaded, the gallery and the remote tree get updated to reflect the new changes.

From: <https://docs.pathomation.com/my/> - **My Pathomation**

Permanent link: [https://docs.pathomation.com/my/doku.php?id=how\\_to\\_add\\_slides&rev=1576433394](https://docs.pathomation.com/my/doku.php?id=how_to_add_slides&rev=1576433394)

Last update: **2019/12/15 21:09**

