


How to add slides?

Adding a new slide can be easily done via the upload container by clicking on “Upload” button located on main header of My Pathomation

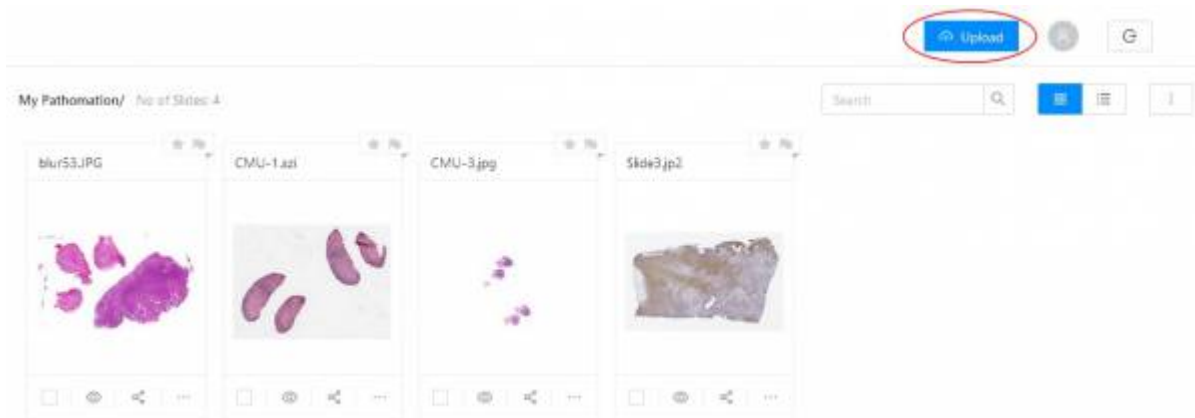


or inside an empty folder



Before starting upload you can change destination folder by clicking on “Upload To” link.

To launch a new upload, you can either :

1. Click on the upload container, a file chooser window pops up, once you select the slide to upload, the upload is started automatically
2. Drag the slide to upload then drop it on the upload container, the upload then starts automatically

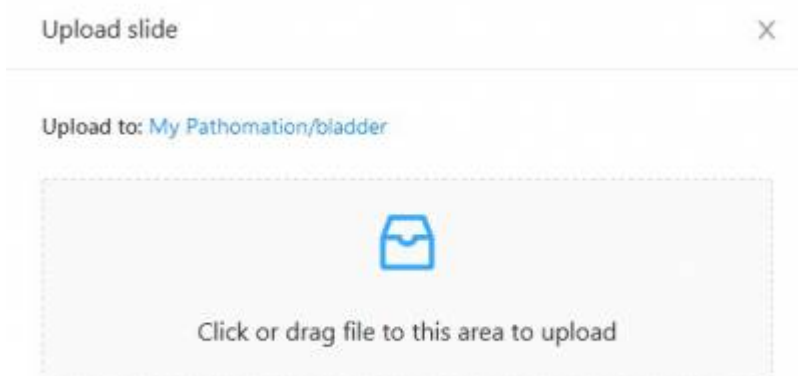

Once the slide is successfully uploaded, the gallery and the remote tree get updated to reflect the new changes.

From:
<https://docs.pathomation.com/my/> - **My Pathomation**

Permanent link:
https://docs.pathomation.com/my/doku.php?id=how_to_add_slides&rev=1644835525

Last update: **2022/02/14 13:45**

