



## How to add slides?

Users have 2 options available to upload slides, via the upload button which is described here or via PMA.start as described in this section <https://docs.pathomation.com/pma.start/doku.php?id=upload>. PMA.start is your best option if you want to manage uploading multiple files at once or require upload of complex multi-file whole slide image formats like .mrxs.

Adding a new slide can be easily done via the upload container by clicking on “Upload” button located on main header of My Pathomation

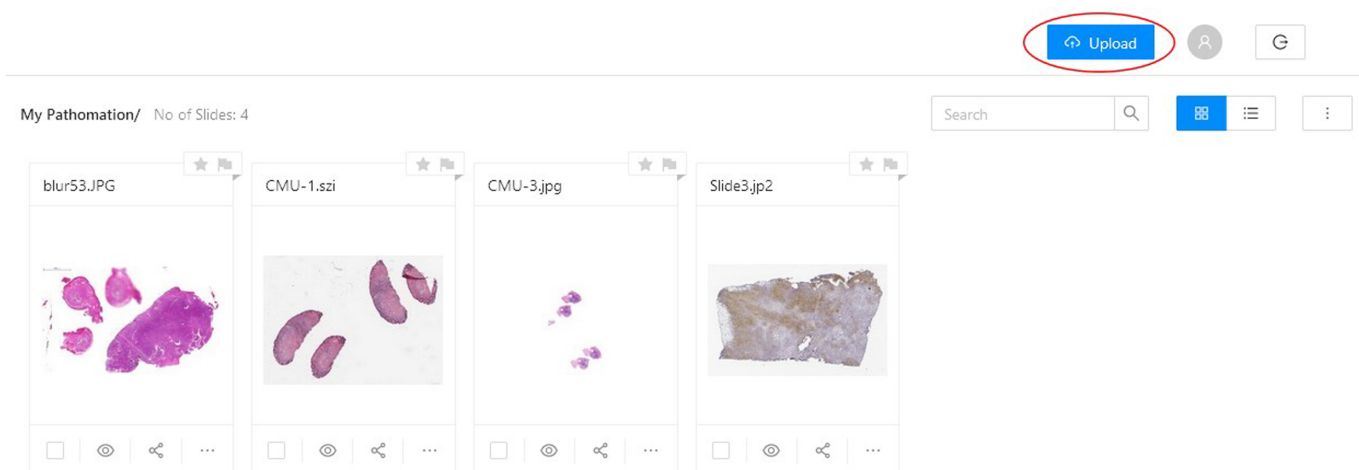

or inside an empty folder

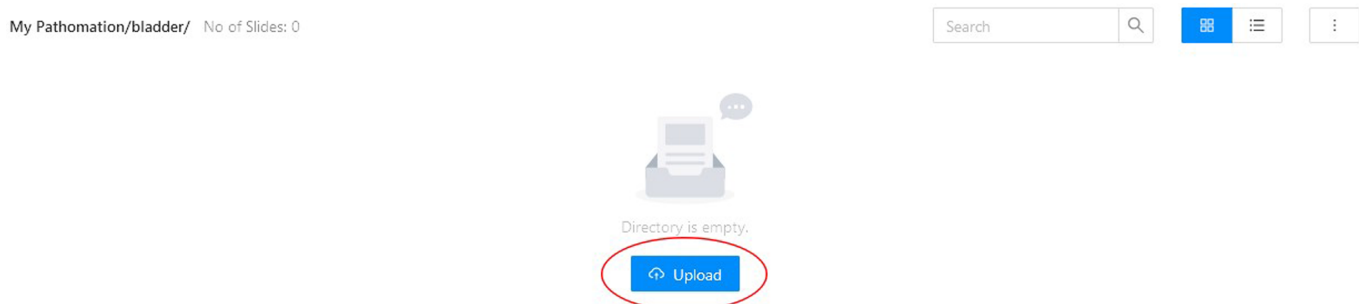

Before starting upload you can change destination folder by clicking on “Upload To” link.

To launch a new upload, you can either :

1. Click on the upload container, a file chooser window pops up, once you select the slide to upload, the upload is started automatically
2. Drag the slide to upload then drop it on the upload container, the upload then starts automatically

## Upload slide



Upload to: [My Pathomation/bladder](#)



Click or drag file to this area to upload

Once the slide is successfully uploaded, the gallery and the remote tree get updated to reflect the new changes.

From:  
<https://docs.pathomation.com/my/> - **My Pathomation**

Permanent link:  
[https://docs.pathomation.com/my/doku.php?id=how\\_to\\_add\\_slides&rev=1661153980](https://docs.pathomation.com/my/doku.php?id=how_to_add_slides&rev=1661153980)

Last update: **2022/08/22 10:39**

