


## How to add slides?

Users have multiple options available to upload slides. You can do this starting from your My Pathomation account or from PMA.start, our free viewer software for desktop use.


When working in your My Pathomation account you have two options:

1. via the upload button in a drag and drop style

Adding a new slide can be easily done via the upload container by clicking on “Upload” button located on main header of My Pathomation

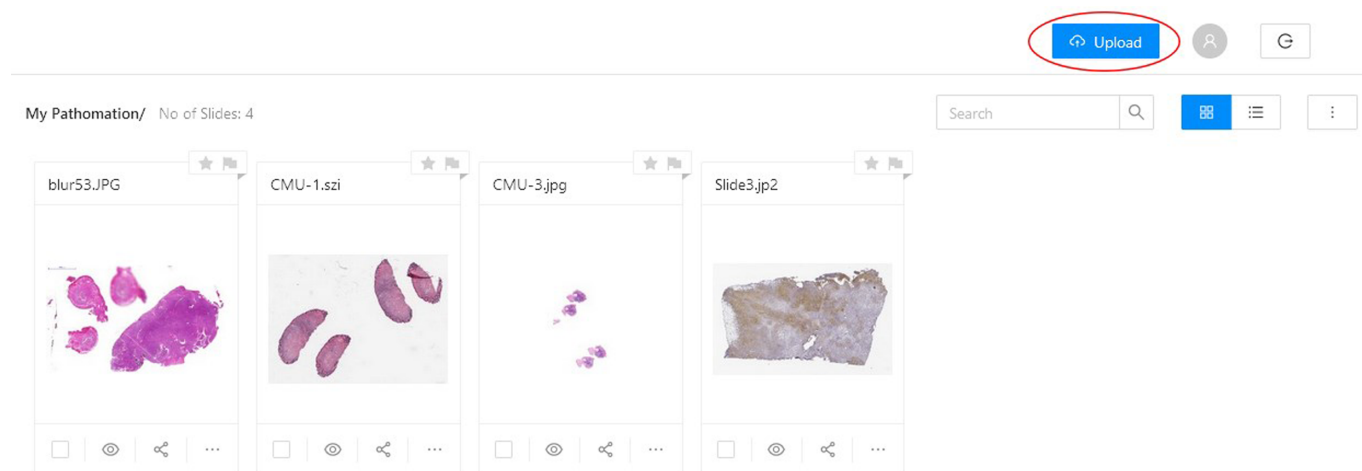

or inside an empty folder

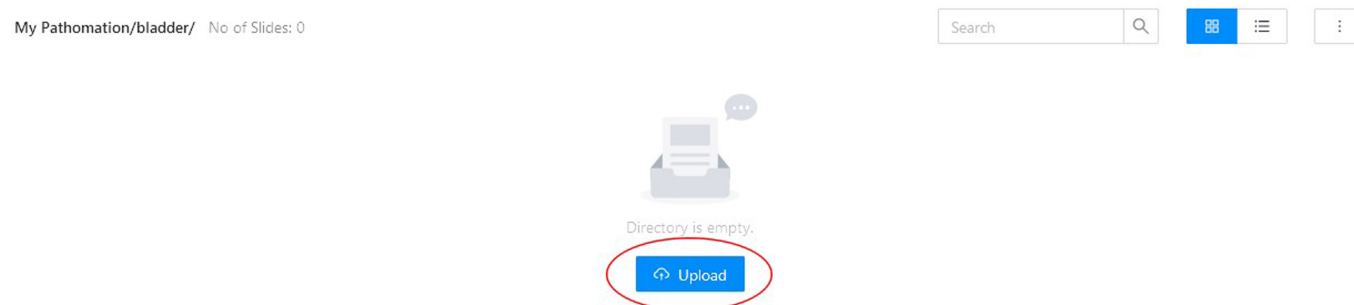

Before starting the upload you can change destination folder by clicking on “Upload To” link.

To launch a new upload, you can either :

1. Click on the upload container, a file chooser window pops up, once you select the slide to upload, the upload is started automatically
2. Drag the slide to upload then drop it on the upload container, the upload then starts automatically

## Upload slide



Upload to: [My Pathomation/bladder](#)




Click or drag file to this area to upload


Once the slide is successfully uploaded, the gallery and the remote tree get updated to reflect the new changes.

### 2. via the 'my pc' section

This approach allows for browsing your pc file folders and then transferring the slides you need to My Pathomation. For this to work you need to install PMA.start, if PMA start is not installed you will get a prompt to install it.

Follow steps 1-4 to upload slides in the my pc section

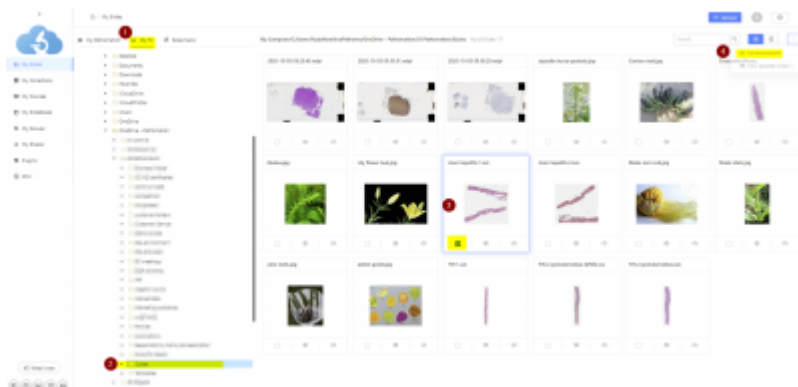

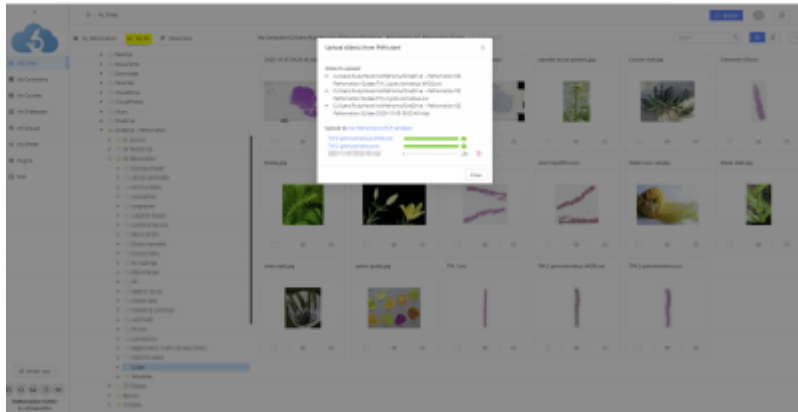

You can also use the file transfer function in PMA.start for uploading to My Pathomation.

This is described in this section <https://docs.pathomation.com/pma.start/doku.php?id=upload>.

PMA.start is your best option if you have slides in various server locations, want to manage uploading multiple files at once or require upload of complex multi-file whole slide image formats like .mrxs.

From: <https://docs.pathomation.com/my/> - **My Pathomation**

Permanent link:  
[https://docs.pathomation.com/my/doku.php?id=how\\_to\\_add\\_slides&rev=1662110170](https://docs.pathomation.com/my/doku.php?id=how_to_add_slides&rev=1662110170)

Last update: **2022/09/02 12:16**

