


# How to host a mini-site?

Before you make a mini-site, make sure you've already uploaded some slides, created at least one case and one collection to add to your mini-site.

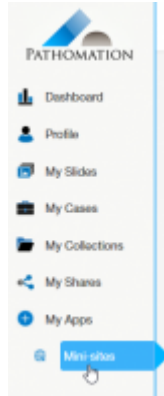

1. Go to “mini-sites” in the sidebar
2. Click on “Add mini-site” on the top right of the page
3. A new window pops up. Here you can enter a name for your mini-site. Once you've entered a name, click “Add”.

From:  
<https://docs.pathomation.com/my/> - **My Pathomation**

Permanent link:  
[https://docs.pathomation.com/my/doku.php?id=how\\_to\\_host\\_a\\_mini-site&rev=1580725355](https://docs.pathomation.com/my/doku.php?id=how_to_host_a_mini-site&rev=1580725355)

Last update: **2020/02/03 13:22**

