


# How to host a mini-site?

Before you make a mini-site, make sure you've already uploaded some slides, created at least one case and one collection to add to your mini-site.

1. Go to “mini-sites” in the sidebar

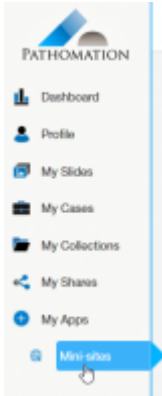

2. Click on “Add mini-site” on the top right of the page



3. A new window pops up. Here you can enter a name for your mini-site. Once you've entered a name, click “Add”.

From:  
<https://docs.pathomation.com/my/> - My Pathomation

Permanent link:  
[https://docs.pathomation.com/my/doku.php?id=how\\_to\\_host\\_a\\_mini-site&rev=1580728783](https://docs.pathomation.com/my/doku.php?id=how_to_host_a_mini-site&rev=1580728783)

Last update: 2020/02/03 14:19

