


# How to host a mini-site?

Before you make a mini-site, make sure you've already uploaded some slides, created at least one case and one collection to add to your mini-site.


1. Go to “mini-sites” in the sidebar

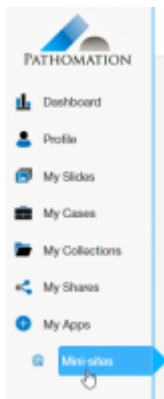

2. Click on “Add mini-site” on the top right of the page



3. A new window pops up. Here you can enter a name for your mini-site and add the collection(s) that you'd like to see in your mini-site. Then click “Add”

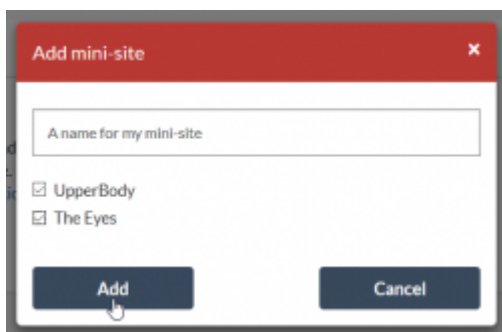

4. Your mini-site is ready! Click on the link to view your mini-site.



Now you can choose to publish your mini-site online by clicking the “publish” button,



Or you can copy the link and send it to whoever you want!

From:  
<https://docs.pathomation.com/my/> - **My Pathomation**

Permanent link:  
[https://docs.pathomation.com/my/doku.php?id=how\\_to\\_host\\_a\\_mini-site&rev=1580731195](https://docs.pathomation.com/my/doku.php?id=how_to_host_a_mini-site&rev=1580731195)

Last update: **2020/02/03 14:59**

