


How to host a slidebox?

Before you make a slidebox, make sure you've already uploaded some slides and created at least one collection to add to your slidebox.


1. Go to "slidebox" in the sidebar

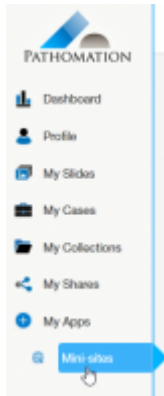

2. Click on "Add slidebox" on the top right of the page



3. A new window pops up. Here you can enter a name for your mini-site and add the collection(s) that you'd like to see in your slidebox. Then click "Add"

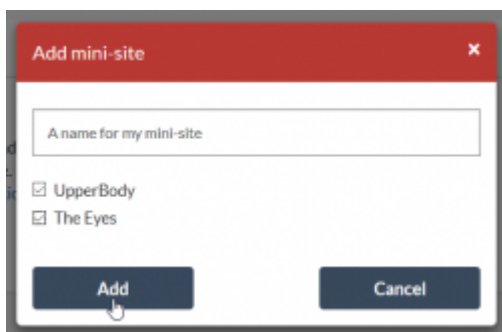

4. Your slidebox is ready! Click on the link to view your brand new slidebox.



Now you can choose to publish your slidebox online by clicking the "publish" button,



Or you can copy the link and send it to whoever you want!

From:
<https://docs.pathomation.com/my/> - **My Pathomation**

Permanent link:
https://docs.pathomation.com/my/doku.php?id=how_to_host_a_slidebox&rev=1585135148

Last update: **2020/03/25 14:19**

