


How to Create an Account in My Pathomation

Start by clicking on the “Register” tab or clicking on <https://my.pathomation.com/register>

In case someone sent you an email invite to join an organization and you have no account yet, you are guided to the registration process after clicking the accept button.

Fill in the form and a personal workspace will be created just for you. Select the hosting location which is closest to you from the drop down menu, this will be the server from which all your slides will be streamed. The hosting location will determine how fast the images will load - the shorter the distance the higher the speed.

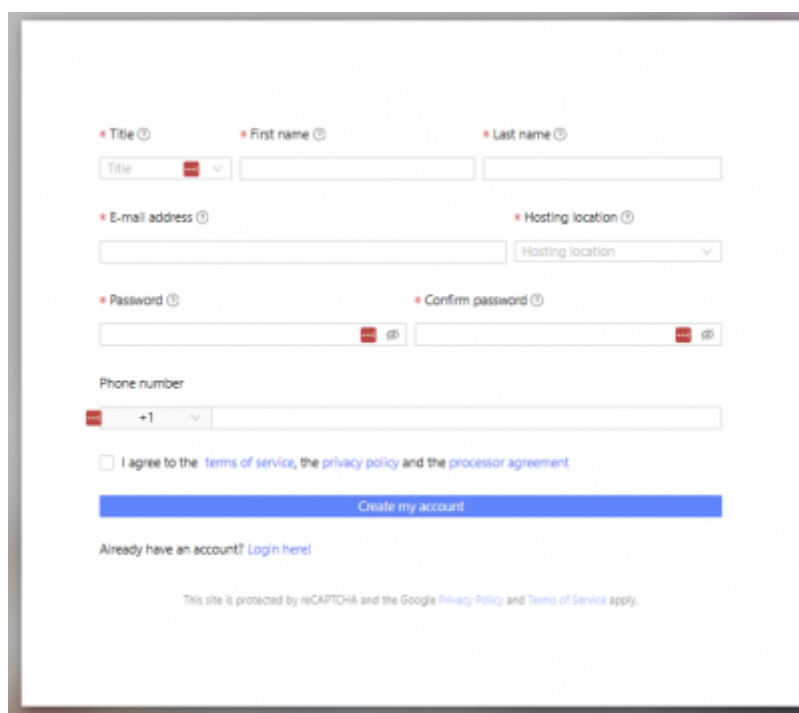

The registration form is titled "Create my account" and is enclosed in a blue border. It contains the following fields and options:

- Title**: A dropdown menu with a red flag icon and a downward arrow.
- First name**: A text input field.
- Last name**: A text input field.
- E-mail address**: A text input field.
- Hosting location**: A dropdown menu with a red flag icon and a downward arrow.
- Password**: A text input field with a red flag icon and a "show" icon (eye).
- Confirm password**: A text input field with a red flag icon and a "show" icon (eye).
- Phone number**: A text input field with a red flag icon and a "+1" dropdown menu.
- Agreement**: A checkbox labeled "I agree to the terms of service, the privacy policy and the processor agreement".
- Create my account**: A blue button.
- Already have an account?**: A link labeled "Login here".
- Footer**: A small text line stating "This site is protected by reCAPTCHA and the Google Privacy Policy and Terms of Service apply."

Then agree to the terms of service, privacy policy and data processor agreement and click “Create my account”

You will now receive an email asking you to confirm the creation of your account. After confirming you can start using your personal workspace in My Pathomation!

In case you do not receive a confirmation email please check your junk folder or ask your IT administrator to add the @pathomation.com to the safe sender list. Still no email confirmation? Contact us at helpdesk@pathomation.com and we'll solve this for you.

Note that you will have the option later to create an organization in which to collaborate and share slides.

From:
<https://docs.pathomation.com/my/> - **My Pathomation**

Permanent link:
https://docs.pathomation.com/my/doku.php?id=how_to_make_an_account_in_my_pathomation

Last update: **2025/09/01 11:47**



