

How to share a case?

1. Go to your cases via "My cases" on the sidebar
2. All your cases are shown in the gallery. On the top right of every case, there is a share button, where you can click on.
3. A new window will pop up!
4. There are different ways to share your case.
 - You can copy the link and share it with others
 - You can embed your slide in your own web content
 - You can generate a QR-code for your slide

From:

<https://docs.pathomation.com/my/> - **My Pathomation**

Permanent link:

https://docs.pathomation.com/my/doku.php?id=how_to_share_a_case&rev=1580732831

Last update: **2020/02/03 15:27**

