

# How to share a case?

1. Go to your cases via “My cases” on the sidebar
2. All your cases are shown in the gallery. On the top right of every case, there is a share button, where you can click on.



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
3. A new window will pop up!

4. There are different ways to share your case.
  - You can copy the link and share it with others
  - You can embed your slide in your own web content
  - You can generate a QR-code for your slide

Choose the option that suits you best. Click next.

5. During this step, you can choose to protect your shared case with a password or an expiration date.

6. Your case is ready to be shared! Copy the URL and send it to others!



From:

<https://docs.pathomation.com/my/> - **My Pathomation**

Permanent link:

[https://docs.pathomation.com/my/doku.php?id=how\\_to\\_share\\_a\\_case&rev=1580733968](https://docs.pathomation.com/my/doku.php?id=how_to_share_a_case&rev=1580733968)

Last update: **2020/02/03 15:46**

