



# How to share a case?


1. Go to your cases via “My cases” on the sidebar

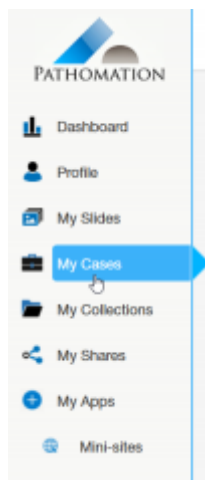

2. All your cases are shown in the gallery. On the top right of every case, there is a share button, where you can click on.

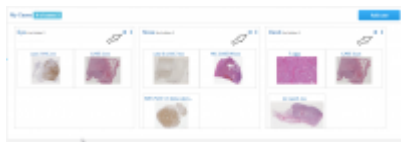

3. A new window will pop up!

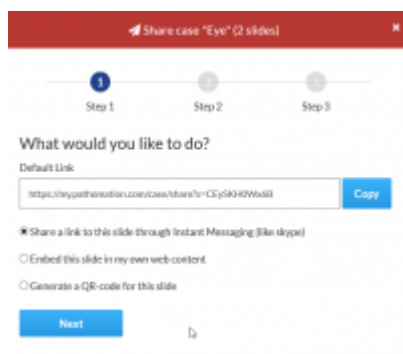

4. There are different ways to share your case.

- You can copy the link and share it with others
- You can embed your slide in your own web content
- You can generate a QR-code for your slide

Choose the option that suits you best. Click next.

5. During this step, you can choose to protect your shared case with a password or an expiration date.

Share case "Eye" (2 slides)

1

2

3

Step 1

Step 2

Step 3

Share a link to this case through Instant Messaging

Select your options

Password protect the link

No

Password

Put an expiration on the link

No

mm/dd/yyyy

Back

Next

6. Your case is ready to be shared! Copy the URL and send it to others!

From:  
<https://docs.pathomation.com/my/> - **My Pathomation**

Permanent link:  
[https://docs.pathomation.com/my/doku.php?id=how\\_to\\_share\\_a\\_case&rev=1580734182](https://docs.pathomation.com/my/doku.php?id=how_to_share_a_case&rev=1580734182)

Last update: **2020/02/03 15:49**

