



# How to share a case?

1. Go to your cases via “My cases” on the sidebar.

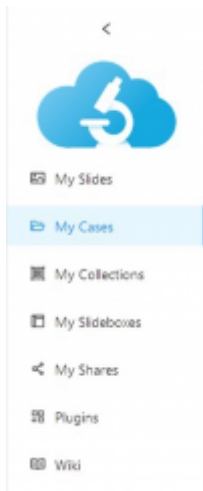

2. All your cases are shown in the gallery. On every case, you can see the share button. Click share button and a new window pops up.

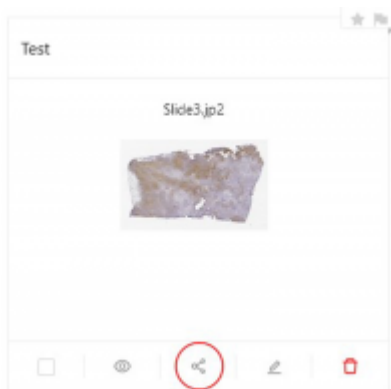


3. There are different ways to share your case.

1. You can copy the link and share it with others
2. You can embed your slide in your own web content
3. You can generate a QR-code for your slide
4. You can email the link of shared slide
5. You can directly send the link of shared slide via various social network service such as *Facebook, Twitter, WhatsApp, Messenger or LinkedIn.*

Also you can change the visibility option of your share. **Unlisted shares** can only be accessed by users with a valid link. **Public shares** can also be accessed by link, but are accessible from public listings too and anyone can view them.


Choose the options that suits your need best.


Share Test X





**Test**  
Created on Feb 10, 2022, 16:33  
Contains 1 slide(s)


Share method Visibility:  Public  Unlisted


  
Link


  
Embed


  
QRcode

  
Email

  
Facebook

  
Twitter


  
WhatsApp

  
Messenger

[ADVANCED \(more options and properties\)](#) [Close](#)


4. By clicking on “ADVANCED (more options and properties)” you can finetune your share. You can password-protect your share, set an expiration date for your share or modify viewport to your needs.


Advanced sharing





**Test**  
Created on Feb 10, 2022, 16:33  
Contains 1 slide(s)


Share method Visibility:  Public  Unlisted


  
Link


  
Embed


  
QRcode


  
Email

  
Facebook

  
Twitter

  
WhatsApp

  
Messenger


  
LinkedIn

**Share options**

Password-protect the share:

Put an expiration on the share:

**Viewport options**

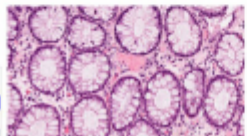

Diagrams  
Default  
**Default 0**  
The name 1:1  
The name 1:2  
The name 1:3

Overview	Visible	Collapsed
Dimensions control	Visible	Collapsed
Barcode	Visible	Collapsed
Rotation control	Visible	Collapsed
File name	Visible	Collapsed
Scale line	Visible	Collapsed
Snapshot button	Visible	
Color adjustments	Visible	

From:  
<https://docs.pathomation.com/my/> - **My Pathomation**

Permanent link:  
[https://docs.pathomation.com/my/doku.php?id=how\\_to\\_share\\_a\\_case&rev=1644842194](https://docs.pathomation.com/my/doku.php?id=how_to_share_a_case&rev=1644842194)

Last update: **2022/02/14 15:36**

