



# How to Share a Collection

1. Go to your collections via “Collections” on the sidebar.

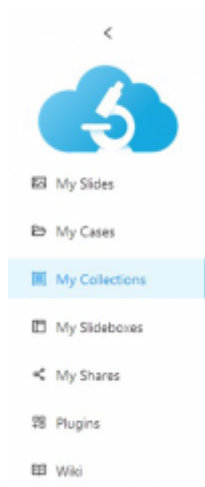

2. All your collections are shown in the gallery. On every collection, you can see the share button. Click the share button to open the share menu.

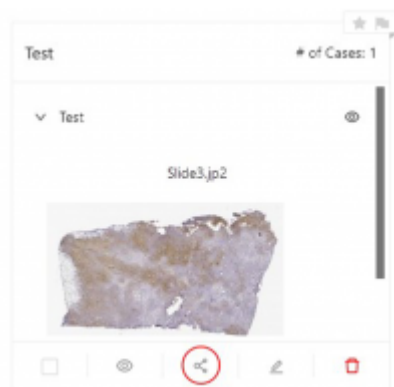

3. There are different ways to share your collection.

1. You can copy the link and share it with others
2. You can embed your collection in your own web content
3. You can generate a QR-code for your collection
4. You can email the link of shared collection
5. You can directly send the link of shared collection via various social network service such as *Facebook, Twitter, WhatsApp, Messenger or LinkedIn*.

You can also change the visibility option of your share. **Unlisted shares** can only be accessed by users with a valid link. **Public shares** can be accessed with direct links, and additionally are accessible from public listings, meaning that anyone can view them.

Choose the option(s) that best suit your needs.

## Share Test



## Test

Created on Feb 10, 2022, 17:37

Contains 1 slide(s) and 1 case(s)

## Share method

Visibility ⓘ: ☐ Public ☒ Unlisted

Link



Embed



QRcode



Email



Facebook



Twitter



WhatsApp




Messenger

[ADVANCED \(more options and properties\)](#)[Close](#)

4. By clicking on “ADVANCED (more options and properties)” you can finetune your share. You can password-protect, set an expiration date, and modify viewport options to your needs.

## Advanced sharing



Test

Created on Feb 10, 2022, 17:37

Contains 1 slide(s) and 1 case(s)

### Share method

Visibility ⓘ: ☐ Public ☒ Unlisted

### Share options

Password-protect the share:

Put an expiration on the share:

### Viewport options

Overview ⓘ: Visible ☒ Collapsed ☐Dimensions control ⓘ: Visible ☒ Collapsed ☐Barcode ⓘ: Visible ☒ Collapsed ☐Rotation control ⓘ: Visible ☒ Collapsed ☐Filename ⓘ: Visible ☐ Collapsed ☐Scale line ⓘ: Visible ☒ Collapsed ☐Snapshot button ⓘ: Visible ☐Color adjustments ⓘ: Visible ☐

From:

<https://docs.pathomation.com/my/> - **My Pathomation**

Permanent link:

[https://docs.pathomation.com/my/doku.php?id=how\\_to\\_share\\_a\\_collection](https://docs.pathomation.com/my/doku.php?id=how_to_share_a_collection)Last update: **2023/11/09 16:46**