



How to share a collection?

1. Go to your collections via “My collections” on the sidebar.

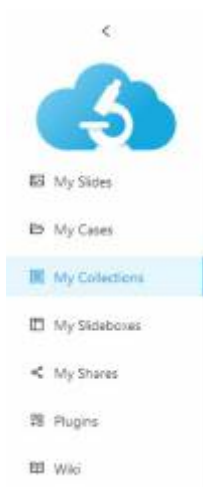

2. All your collections are shown in the gallery. On every collection, you can see the share button. Click share button and a new window pops up.

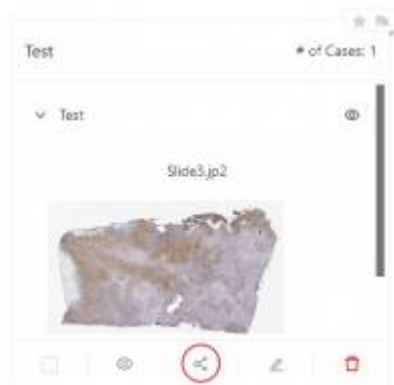

3. There are different ways to share your collection.

1. You can copy the link and share it with others
2. You can embed your slide in your own web content
3. You can generate a QR-code for your slide
4. You can email the link of shared slide
5. You can directly send the link of shared slide via various social network service such as *Facebook, Twitter, WhatsApp, Messenger or LinkedIn.*

Also you can change the visibility option of your share. **Unlisted shares** can only be accessed by users with a valid link. **Public shares** can also be accessed by link, but are accessible from public listings too and anyone can view them.

Choose the options that suits your need best.

Share Test

Test

Created on Feb 10, 2022, 17:37

Contains 1 slide(s) and 1 case(s)

Share method

Visibility: Public Unlisted

Link Embed QRcode Email Facebook Twitter WhatsApp Messenger

ADVANCED (more options and properties) Close

4. By clicking on “ADVANCED (more options and properties)” you can finetune your share. You can password-protect your share, set an expiration date for your share or modify viewport to your needs.

Advanced sharing

Test

Created on Feb 10, 2022, 17:37

Contains 1 slide(s) and 1 case(s)

Share method

Visibility: Public Unlisted

Link Embed QRcode Email Facebook Twitter WhatsApp Messenger LinkedIn

Share options

Password-protect the share: Password

Put an expiration on the share: Select date

Viewport options

Overview: Visible Collapsed

Dimensions control: Visible Collapsed

Barcode: Visible Collapsed

Rotation control: Visible Collapsed

Fullscreen: Visible Collapsed

Scale line: Visible Collapsed

Spotlight button: Visible

Color adjustments: Visible

Share & Print

From: <https://docs.pathomation.com/my/> - **My Pathomation**

Permanent link: https://docs.pathomation.com/my/doku.php?id=how_to_share_a_collection&rev=1644834960

Last update: **2022/02/14 13:36**

