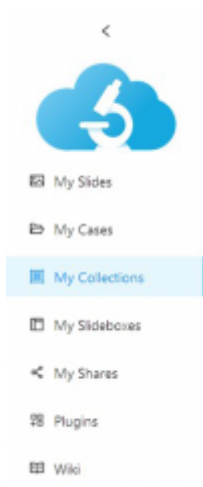
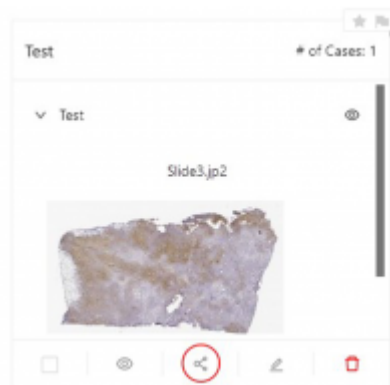


How to share a collection?

1. Go to your collections via “My collections” on the sidebar.



2. All your collections are shown in the gallery. On every collection, you can see the share button. Click share button and a new window pops up.




3. There are different ways to share your collection.

1. You can copy the link and share it with others
2. You can embed your slide in your own web content
3. You can generate a QR-code for your slide
4. You can email the link of shared slide
5. You can directly send the link of shared slide via various social network service such as *Facebook, Twitter, WhatsApp, Messenger or LinkedIn.*

Also you can change the visibility option of your share. **Unlisted shares** can only be accessed by users with a valid link. **Public shares** can also be accessed by link, but are accessible from public listings too and anyone can view them.


Choose the options that suits your need best.


Share Test X





Test
Created on Feb 10, 2022, 17:37
Contains 1 slide(s) and 1 case(s)


Share method Visibility: Public Unlisted



Link



Embed



QRcode


Email


Facebook


Twitter



WhatsApp


Messenger

[ADVANCED \(more options and properties\)](#) [Close](#)


4. By clicking on “ADVANCED (more options and properties)” you can finetune your share. You can password-protect your share, set an expiration date for your share or modify viewport to your needs.


Advanced sharing





Test
Created on Feb 10, 2022, 17:37
Contains 1 slide(s) and 1 case(s)


Share method Visibility: Public Unlisted



Link



Embed



QRcode



Email


Facebook


Twitter


WhatsApp


Messenger

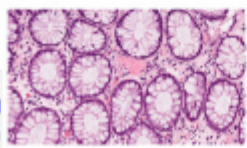

LinkedIn

Share options

Password-protect the share:

Put an expiration on the share:

Viewport options



Themes
Default
Default 2

File Name 1: 2
Default 1: 2

<input checked="" type="checkbox"/>	Overview	<input type="radio"/> Visible	<input checked="" type="checkbox"/> Collapsed
<input checked="" type="checkbox"/>	Dimensions control	<input type="radio"/> Visible	<input checked="" type="checkbox"/> Collapsed
<input checked="" type="checkbox"/>	Barcode	<input type="radio"/> Visible	<input checked="" type="checkbox"/> Collapsed
<input checked="" type="checkbox"/>	Rotation control	<input type="radio"/> Visible	<input checked="" type="checkbox"/> Collapsed
<input checked="" type="checkbox"/>	Filename	<input type="radio"/> Visible	<input checked="" type="checkbox"/> Collapsed
<input checked="" type="checkbox"/>	Scale line	<input type="radio"/> Visible	<input checked="" type="checkbox"/> Collapsed
<input type="checkbox"/>	Snapshot button	<input type="radio"/> Visible	<input type="checkbox"/> Collapsed
<input checked="" type="checkbox"/>	Color adjustments	<input type="radio"/> Visible	<input type="checkbox"/> Collapsed

From:
<https://docs.pathomation.com/my/> - **My Pathomation**

Permanent link:
https://docs.pathomation.com/my/doku.php?id=how_to_share_a_collection&rev=1644842206

Last update: **2022/02/14 15:36**

