


How to share a slide?

1. Go to your slides via “My slides” on the sidebar.

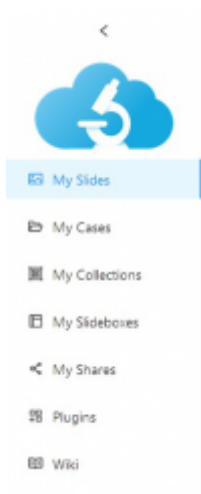

2. All your slides are shown in the gallery. On every slide, you can see the share button. Click share button and a new window pops up.




3. There are different ways to share your slide.

1. You can copy the link and share it with others
2. You can embed your slide in your own web content
3. You can generate a QR-code for your slide
4. You can email the link of shared slide
5. You can directly send the link of shared slide via various social network service such as *Facebook, Twitter, WhatsApp, Messenger or LinkedIn*. Watch our explanatory video here https://youtu.be/Y73I7nm_8d8


Also you can change the visibility option of your share. **Unlisted shares** can only be accessed by users with a valid link. **Public shares** can also be accessed by link, but are accessible from public listings too and anyone can view them.


Choose the options that suits your need best.


Share Slide3.jp2 ✕


My Pathomation/Slide3.jp2


Share method Visibility ⓘ: Public Unlisted



Link



Embed



QRcode


Email


Facebook


Twitter



WhatsApp


Messenger

[ADVANCED \(more options and properties\)](#)[Close](#)

4. By clicking on “ADVANCED (more options and properties)” you can finetune your share. You can password-protect your share, set an expiration date for your share or modify viewport to your needs.

Advanced sharing

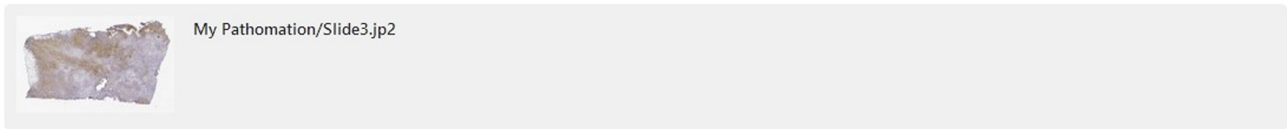

Share method

Visibility : Public Unlisted


- Link
- Embed
- QRcode
- Email
- Facebook
- Twitter
- WhatsApp
- Messenger
- LinkedIn

Share options

Password-protect the share:

Put an expiration on the share:

Viewport options

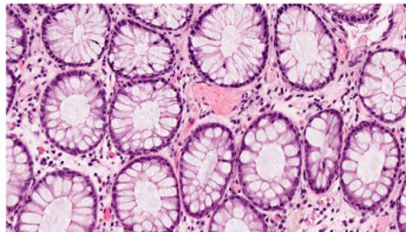

Channels

- Default
- Default

Time frame: 1 / 2

Z-stack: 2 / 2



Overview : Visible Collapsed

Dimensions control : Visible Collapsed



Barcode : Visible Collapsed

Rotation control : Visible Collapsed

Filename : Visible Collapsed

Scale line : Visible Collapsed

Snapshot button : Visible



Color adjustments : Visible

From: <https://docs.pathomation.com/my/> - **My Pathomation**

Permanent link: https://docs.pathomation.com/my/doku.php?id=how_to_share_a_slide&rev=1675356312

Last update: 2023/02/02 19:45

