


# How to share a slide?

1. Go to your slides via “My slides” on the sidebar.

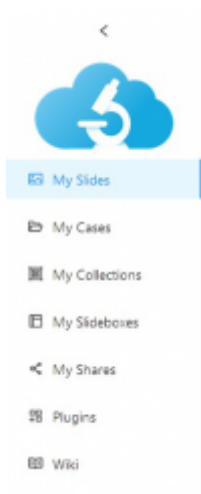

2. The standard view setting is list view, in this view you need to click the ... icon next to the slide and the share button will appear. If you work in the gallery view you can see the share button next to the slide. Click the share button and a new window pops up.



3. There are different ways to share your slide.

1. You can copy the link and share it with others
2. You can embed your slide in your own web content
3. You can generate a QR-code for your slide
4. You can email the link of shared slide
5. You can directly send the link of shared slide via various social network service such as *Facebook, Twitter, WhatsApp, Messenger or LinkedIn*. Watch our explanatory video here [https://youtu.be/Y73I7nm\\_8d8](https://youtu.be/Y73I7nm_8d8)


Also you can change the visibility option of your share. **Unlisted shares** can only be accessed by users with a valid link. **Public shares** can also be accessed by link, but are accessible from public

listings too and anyone can view them.


Choose the options that suits your need best.


### Share Slide3.jp2 X


---


My Pathomation/Slide3.jp2


Share method Visibility ⓘ:  Public  Unlisted


  
Link


  
Embed


  
QRcode

  
Email

  
Facebook

  
Twitter

  
WhatsApp


  
Messenger

---

[ADVANCED \(more options and properties\)](#)[Close](#)

4. By clicking on “ADVANCED (more options and properties)” you can finetune your share. You can password-protect your share, set an expiration date for your share or modify viewport to your needs.

### Advanced sharing

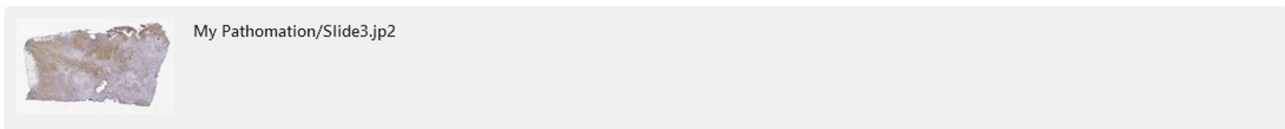

Share method

Visibility:  Public  Unlisted

- Link
- Embed
- QRcode
- Email
- Facebook
- Twitter
- WhatsApp
- Messenger
- LinkedIn

### Share options

Password-protect the share:

Put an expiration on the share:

### Viewport options

Channels

Default

Time frame: 1 / 2

Z-stack: 2 / 2

- Overview: Visible  Collapsed
- Dimensions control: Visible  Collapsed
- Barcode: Visible  Collapsed
- Rotation control: Visible  Collapsed
- Filename: Visible  Collapsed
- Scale line: Visible  Collapsed
- Snapshot button: Visible
- Color adjustments: Visible

From:  
<https://docs.pathomation.com/my/> - **My Pathomation**

Permanent link:  
[https://docs.pathomation.com/my/doku.php?id=how\\_to\\_share\\_a\\_slide&rev=1677138519](https://docs.pathomation.com/my/doku.php?id=how_to_share_a_slide&rev=1677138519)

Last update: 2023/02/23 10:48

