


# How to Share a Slidebox

1. Go to your slideboxes via "Slideboxes" on the sidebar.

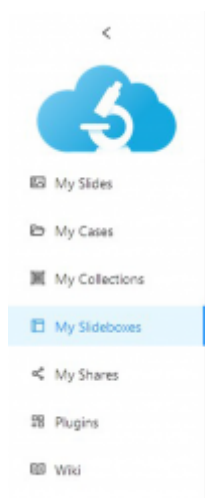

2. All your slideboxes are shown in the gallery. On every slidebox, you can see the share button. Click the share button to open the share menu.



3. There are different ways to share your slidebox.

1. You can copy the link to your slidebox and share it with others
2. You can embed your slidebox in your own web content
3. You can generate a QR-code for your slidebox
4. You can email the link of shared slidebox
5. You can directly send the link of shared slidebox via various social network service such as *Facebook, Twitter, WhatsApp, Messenger or LinkedIn*.

Also you can change the visibility option of your share. **Unlisted shares** can only be accessed by users with a valid link. **Public shares** can be accessed with direct links, and additionally are accessible from public listings, meaning that anyone can view them.

Choose the options that suits your need best.

## Share Test



## Test

Created on Feb 14, 2022, 11:23

Contains 1 slide(s), 1 case(s) and 1 collection(s)

## Share method

Visibility ⓘ: ☐ Public ☒ Unlisted

Link



Embed



QRcode



Email



Facebook



Twitter



WhatsApp




Messenger

[ADVANCED \(more options and properties\)](#)[Close](#)

4. By clicking on “ADVANCED (more options and properties)” you can finetune your share. You can password-protect, set an expiration date, and modify viewport options to your needs.

## Advanced sharing



**Test**


Created on Feb 14, 2022, 11:23

Contains 1 slide(s), 1 case(s) and 1 collection(s)

### Share method

Visibility ⓘ: ☐ Public ☒ Unlisted

### Viewport options

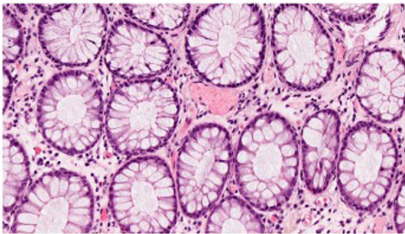

Channels



Default ☒

Default ☒

Time frame: 1 / 2

Z-stack: 2 / 2

1000 µm



Overview ⓘ: Visible ☒ Collapsed ☐

Dimensions control ⓘ: Visible ☒ Collapsed ☐

Barcode ⓘ: Visible ☒ Collapsed ☐

Rotation control ⓘ: Visible ☒ Collapsed ☐

Filename ⓘ: Visible ☐ Collapsed ☐

Scale line ⓘ: Visible ☒ Collapsed ☐

Snapshot button ⓘ: Visible ☐

Color adjustments ⓘ: Visible ☐

Zoom slider ⓘ: Visible ☒

From:

<https://docs.pathomation.com/my/> - **My Pathomation**

Permanent link:

[https://docs.pathomation.com/my/doku.php?id=how\\_to\\_share\\_a\\_slidebox](https://docs.pathomation.com/my/doku.php?id=how_to_share_a_slidebox)Last update: **2023/11/09 16:49**