


# How to share a slidebox?

1. Go to your slideboxes via “My slideboxes” on the sidebar.

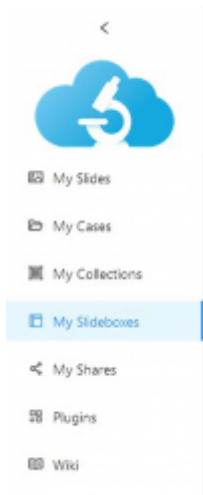

2. All your slideboxes are shown in the gallery. On every slidebox, you can see the share button. Click share button and a new window pops up.




3. There are different ways to share your slidebox.

1. You can copy the link and share it with others
2. You can embed your slide in your own web content
3. You can generate a QR-code for your slide
4. You can email the link of shared slide
5. You can directly send the link of shared slide via various social network service such as *Facebook, Twitter, WhatsApp, Messenger or LinkedIn.*

Also you can change the visibility option of your share. **Unlisted shares** can only be accessed by users with a valid link. **Public shares** can also be accessed by link, but are accessible from public listings too and anyone can view them.


Choose the options that suits your need best.


Share Test X





**Test**  
Created on Feb 14, 2022, 11:23  
Contains 1 slide(s), 1 case(s) and 1 collection(s)


Share method Visibility:  Public  Unlisted


  
Link


  
Embed


  
QRcode

  
Email

  
Facebook

  
Twitter

  
WhatsApp

  
Messenger

[ADVANCED \(more options and properties\)](#) [Close](#)

4. By clicking on “ADVANCED (more options and properties)” you can finetune viewport of your share.

Advanced sharing



**Test**  
Created on Feb 14, 2022, 11:23  
Contains 1 slide(s), 1 case(s) and 1 collection(s)

Share method Visibility:  Public  Unlisted

  
Link

  
Embed

  
QRcode

  
Email

  
Facebook

  
Twitter

  
WhatsApp

  
Messenger

  
LinkedIn

Viewport options



Thumbnail of the slide viewer interface showing a grid of histology slides.

- Overview: Visible  Collapsed
- Dimensions control: Visible  Collapsed
- Barcode: Visible  Collapsed
- Rotation control: Visible  Collapsed
- Filename: Visible  Collapsed
- Scale line: Visible  Collapsed
- Snapshot button: Visible
- Color adjustments: Visible
- Zoom slider: Visible

From:  
<https://docs.pathomation.com/my/> - My Pathomation

Permanent link:  
[https://docs.pathomation.com/my/doku.php?id=how\\_to\\_share\\_a\\_slidebox&rev=1644842215](https://docs.pathomation.com/my/doku.php?id=how_to_share_a_slidebox&rev=1644842215)

Last update: **2022/02/14 15:36**

