


# How to annotate slides?

My Pathomation comes with a set of tools to annotate and draw on your slides have been updated. The available tools are listed here:

- 1) Straight line & measurement tool
- 2) Arrow & text tool
- 3) Circle tool
- 4) Square tool
- 5) Freehand line and measurement tool
- 6) Closed freehand tool
- 7) Text tool

An illustration of the results is shown here

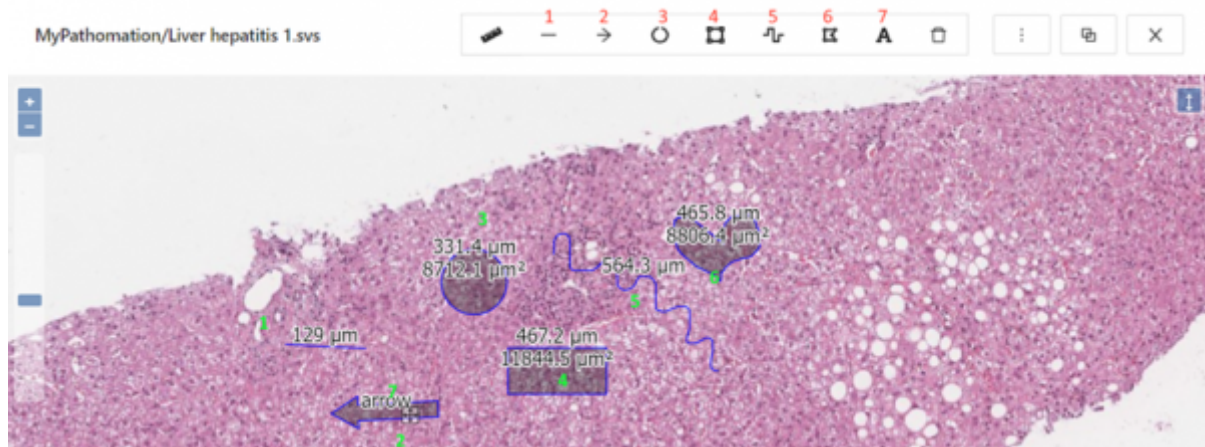

The annotations you make on a slide will persist if you include this slide in a collection, slide box or course

The line tool will show you the length of the line, the area tools will show the surface area inside the marked area. If you want to make a quick series of line distance measurements that do not need to be stored you can use the measurement tool, the use is explained in this video <https://youtu.be/AfVNdafwNMw>

From:  
<https://docs.pathomation.com/my/> - **My Pathomation**

Permanent link:  
[https://docs.pathomation.com/my/doku.php?id=how\\_to\\_use\\_annotations&rev=1699286064](https://docs.pathomation.com/my/doku.php?id=how_to_use_annotations&rev=1699286064)

Last update: **2023/11/06 18:54**

