

# Integrating Your Interactive Slides into Powerpoint.

It's very easy to boost your powerpoint quality with [sharing a slide](#) with a QR code that people can scan and interact with on their phones and laptops.

## But we can go one step further

In this article you will learn how to integrate a fully interactive slide directly into a powerpoint slide that you can navigate during your talk without ever leaving the presentation.

## Video Guide


This helpful video explains how to get started:

<https://www.youtube.com/watch?v=hTghEqccVBk&t=5s>

Or if you'd prefer written instructions:


## Walkthrough

**Installing Web Viewer Plugin** 1) First off you'll need Web Viewer plugin for Microsoft Powerpoint. You can search and download this from: - Powerpoint via the 'Insert' tab and clicking 'Get Add Ins'

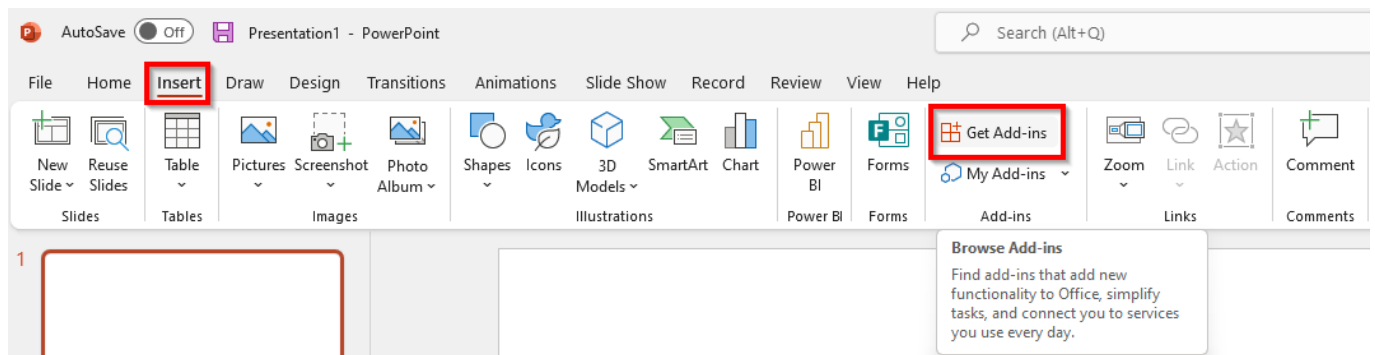

- Or the online office store

<https://appssource.microsoft.com/en-us/product/office/WA104295828?tab=Overview>

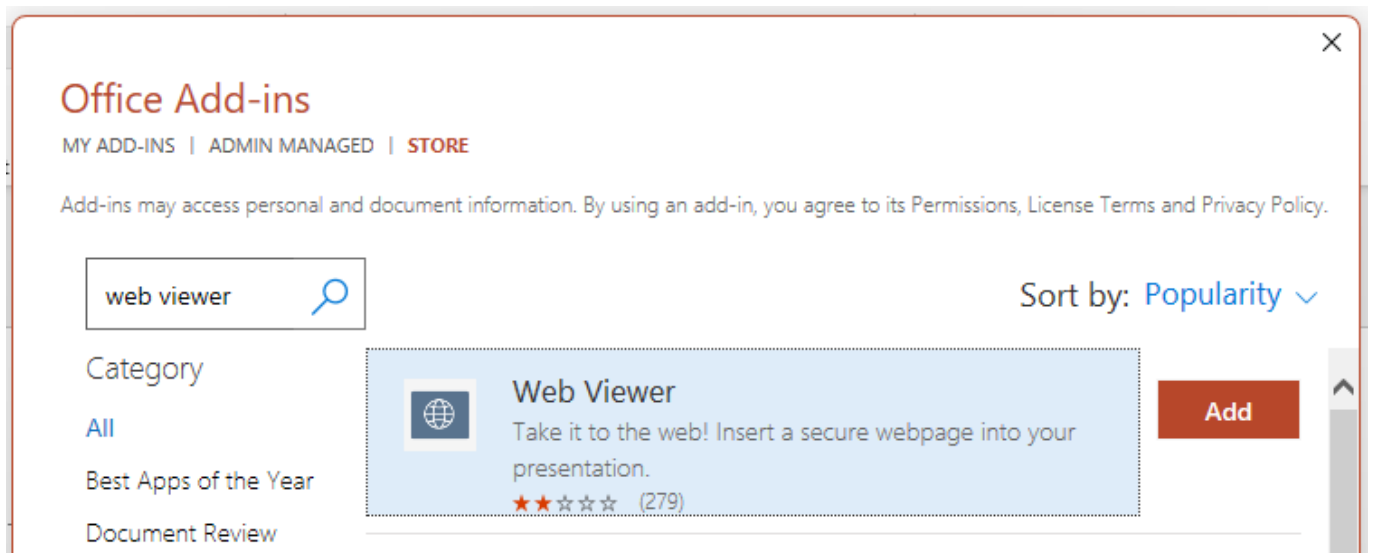

The screenshot shows the 'Office Add-ins' interface. At the top, it says 'Office Add-ins' with sub-links for 'MY ADD-INS', 'ADMIN MANAGED', and 'STORE'. Below this is a disclaimer: 'Add-ins may access personal and document information. By using an add-in, you agree to its Permissions, License Terms and Privacy Policy.' A search bar contains the text 'web viewer'. To the right, it says 'Sort by: Popularity'. On the left, there is a 'Category' list with options: 'All', 'Best Apps of the Year', and 'Document Review'. The main area displays the 'Web Viewer' add-in card, which includes a globe icon, the title 'Web Viewer', the description 'Take it to the web! Insert a secure webpage into your presentation.', a star rating of four stars, and '(279)' reviews. A red 'Add' button is visible to the right of the card.

- 2) Click Add, and follow the installation steps
- 3) Restart Powerpoint to complete the installation

### Inserting Your Slide



From: <https://docs.pathomation.com/my/> - My Pathomation

Permanent link: [https://docs.pathomation.com/my/doku.php?id=integrating\\_an\\_interactive\\_slide\\_into\\_powerpoint&rev=1706698778](https://docs.pathomation.com/my/doku.php?id=integrating_an_interactive_slide_into_powerpoint&rev=1706698778)

Last update: 2024/01/31 13:59

