

Integrating Your Interactive Slides into Powerpoint.

It's very easy to boost your powerpoint quality with [sharing a slide](#) with a QR code that people can scan and interact with on their phones and laptops.

But we can go one step further

In this article you will learn how to integrate a fully interactive slide directly into a powerpoint slide that you can navigate during your talk without ever leaving the presentation.

Video Guide


This helpful video explains how to get started:

<https://www.youtube.com/watch?v=hTghEqccVBk&t=5s>

Or if you'd prefer written instructions:


Walkthrough

Installing Web Viewer Plugin 1) First off you'll need Web Viewer plugin for Microsoft Powerpoint. You can search and download this from: - Powerpoint via the 'Insert' tab and clicking 'Get Add Ins'

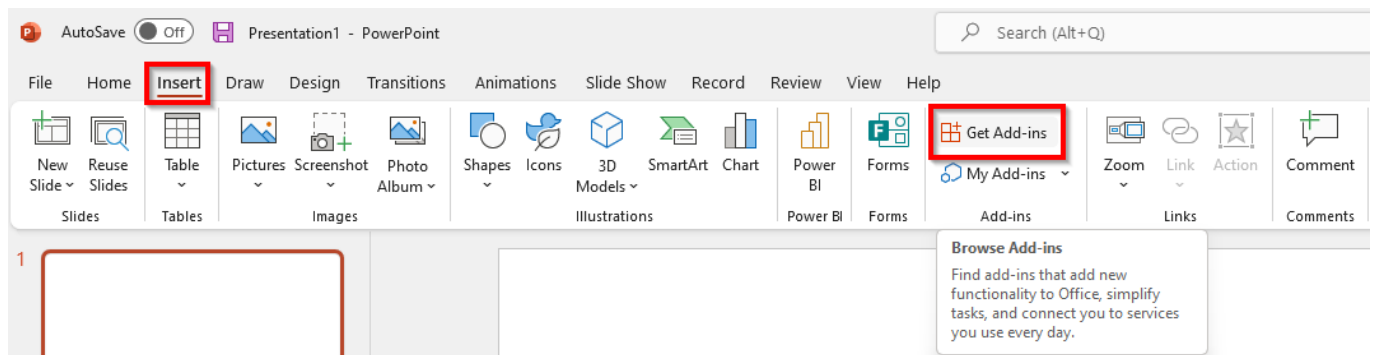

- Or the online office store

<https://appsourc.microsoft.com/en-us/product/office/WA104295828?tab=Overview>

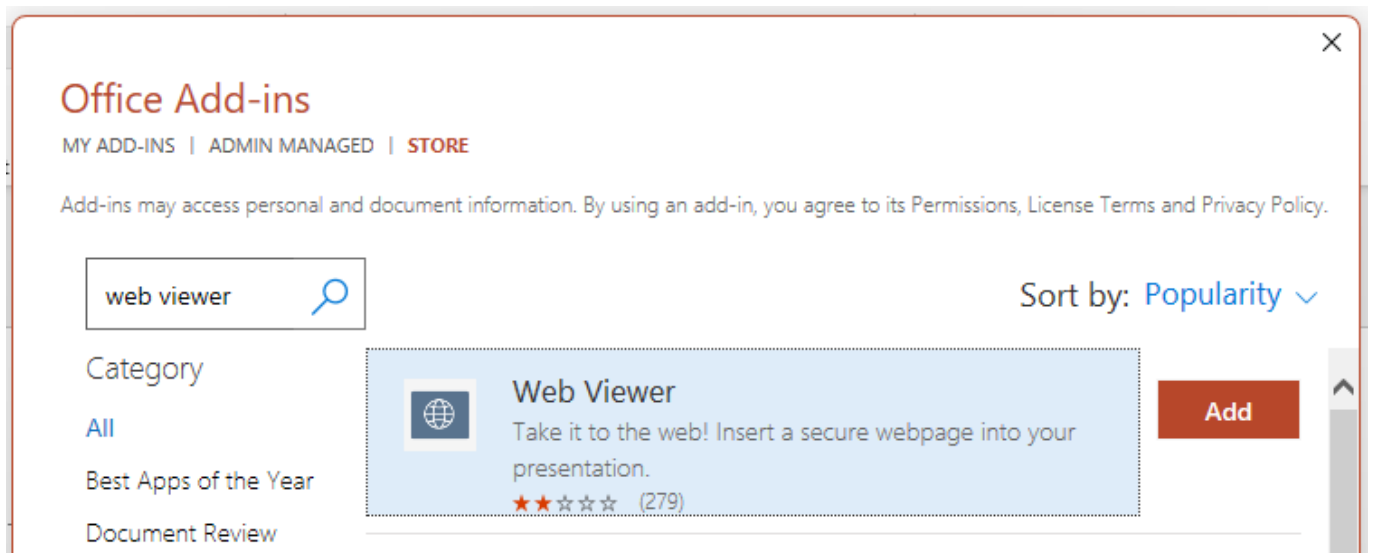

The screenshot shows the 'Office Add-ins' interface. At the top, it says 'Office Add-ins' and 'MY ADD-INS | ADMIN MANAGED | STORE'. Below that, a warning states: 'Add-ins may access personal and document information. By using an add-in, you agree to its Permissions, License Terms and Privacy Policy.' A search bar contains the text 'web viewer'. To the right, it says 'Sort by: Popularity'. On the left, there is a 'Category' list with options: 'All', 'Best Apps of the Year', and 'Document Review'. The main area displays the 'Web Viewer' add-in with a globe icon, the text 'Web Viewer', a description 'Take it to the web! Insert a secure webpage into your presentation.', and a rating of four stars with '(279)' reviews. A red 'Add' button is visible to the right of the add-in card.

- 2) Click Add, and follow the installation steps
- 3) Restart Powerpoint to complete the installation

Inserting Your Slide



From: <https://docs.pathomation.com/my/> - My Pathomation

Permanent link: https://docs.pathomation.com/my/doku.php?id=integrating_an_interactive_slide_into_powerpoint&rev=1706698778

Last update: 2024/01/31 13:59

