



## Interpreting AI results

After completion of an image analysis AI request the results will be available in the form of annotations in the viewer and as a set of results.

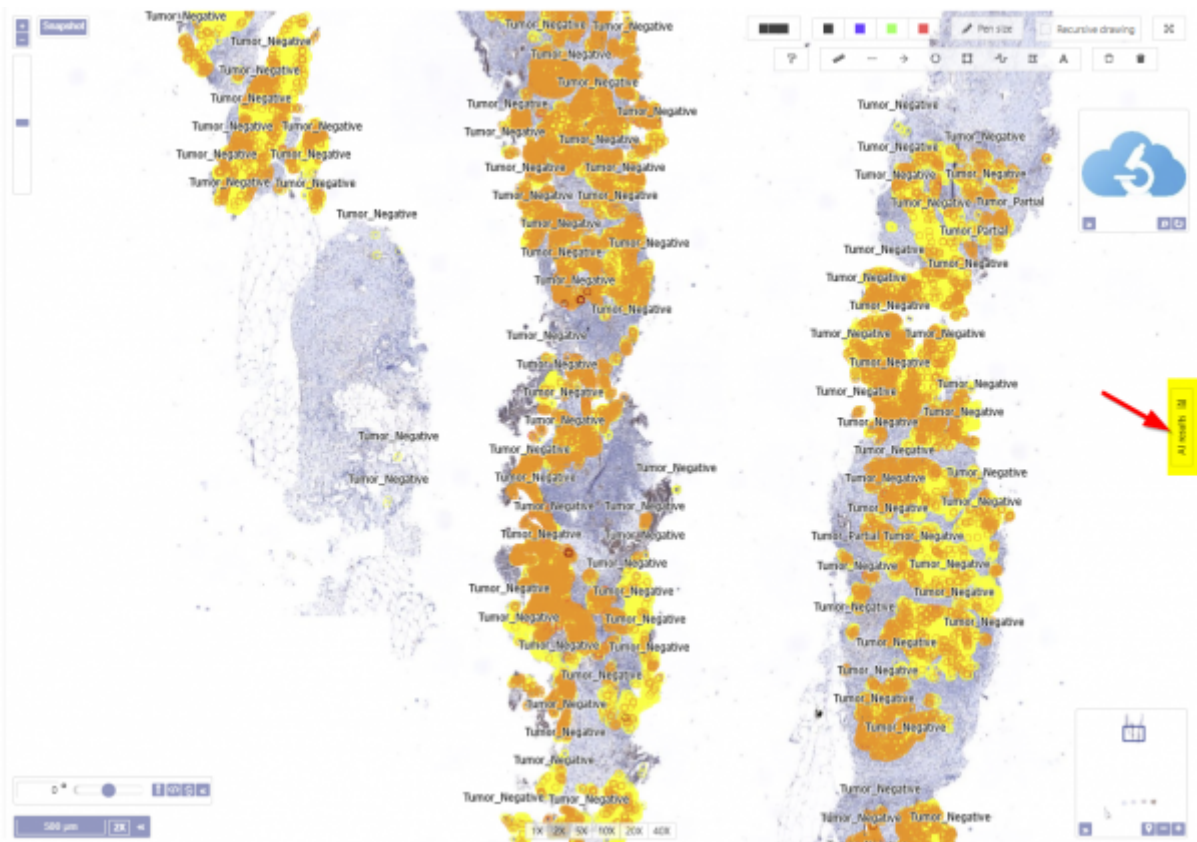


If annotations are text labelled by the algorithm the text will appear in the viewer and scale with the zoom level. You can hide the text with the label button for better visibility of the image.


Click on the AI results tab to show the interpretation done by the algorithm.

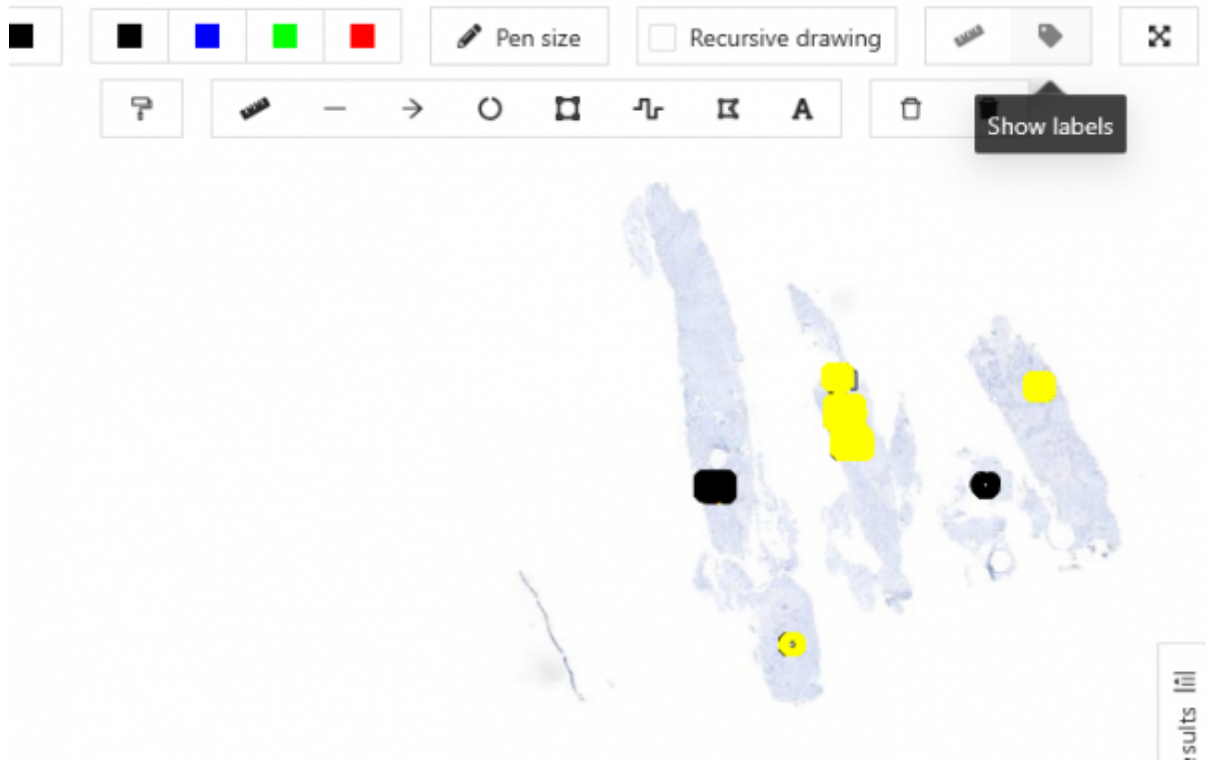


The sidebar AI result is available to open on the right hand side of the viewer. The returned results are available including number of findings and a score or number if relevant. The annotations are shown as a menu per category. Users can change the colour of a category if needed. Please note that when you mouse over a category this triggers a highlight of that annotation category in the viewer window.

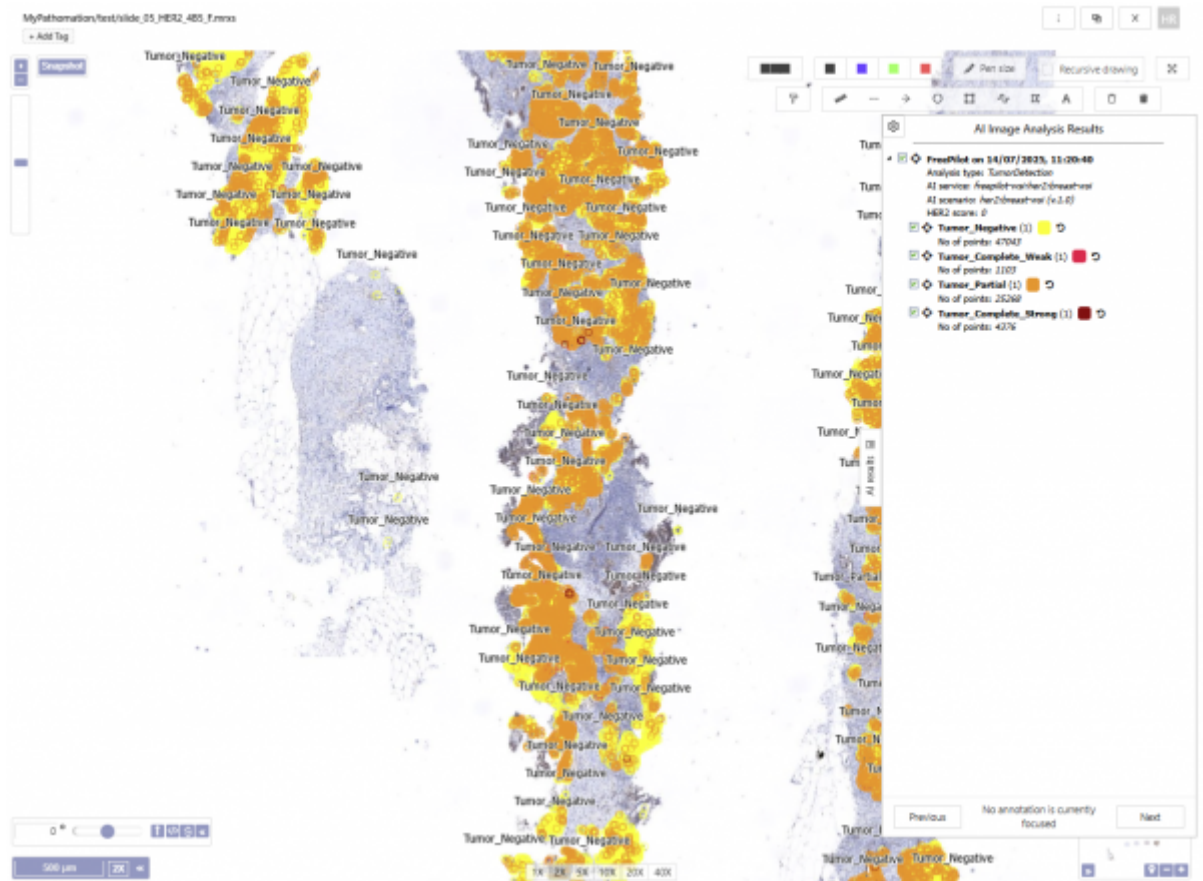

It is also possible to deselect and reselect all categories

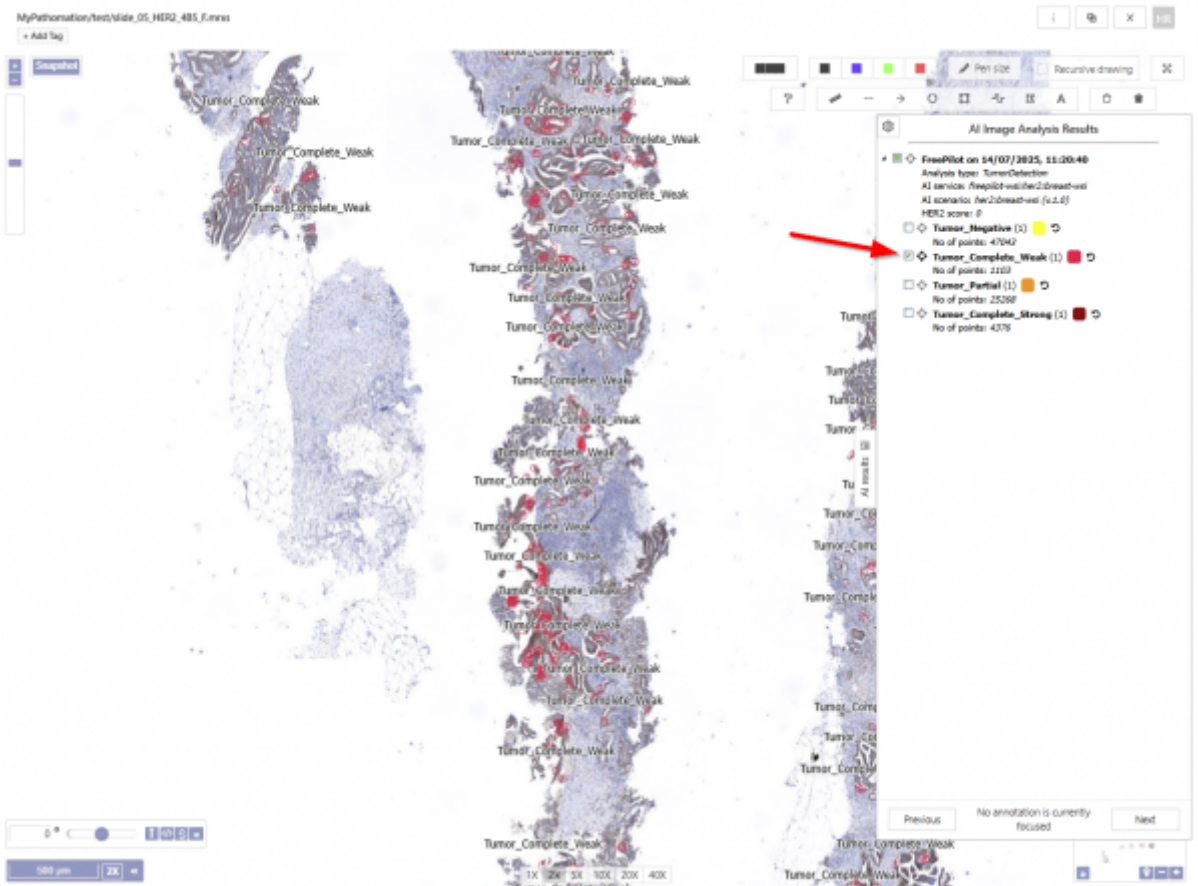

Textual AI results can be copied from the result pane for use in reports and databases.

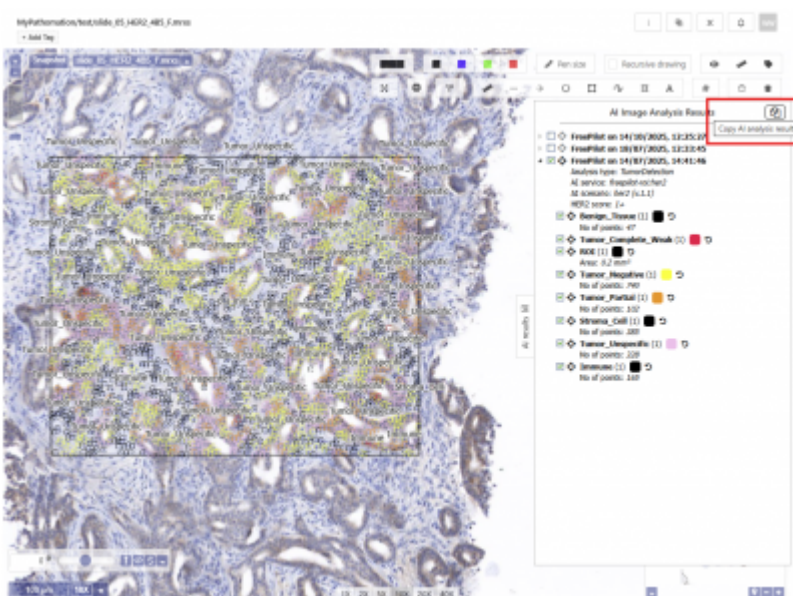

From:

<https://docs.pathomation.com/my/> - **My Pathomation**

Permanent link:

[https://docs.pathomation.com/my/doku.php?id=interpreting\\_ai\\_results  
&rev=1774260661](https://docs.pathomation.com/my/doku.php?id=interpreting_ai_results&rev=1774260661)

Last update: **2026/03/23 13:11**

